

Getting Started for

# Parents & Caregivers



The ableEXPERIENCE includes a hands-on opportunity with your QuickTalker Freestyle. Using the device allows everyone to learn and determine best outcomes.

There is no obligation to complete the funding process and we have a team that will work with you to retrieve the device at no cost.



## Next Steps

- **Hands-on Opportunity:** Make sure your loved one attends their speech therapy sessions so that their speech therapist can work with them and the device.
- **Insurance Decision:** AbleNet will collect and submit the required documentation to your loved one's insurance for their final decision.



## Quick Start Guide

Your Device



QuickTalker Freestyle



QuickTalker Freestyle Mini

- A** On/Off Button
- B** Volume Buttons
- C** Charging Port

## Getting Started

1. Press and hold the on/off button for 3-5 seconds to turn your device on.
2. Tap the speech app icon to start using your speech app.
3. Charge your device overnight when it's not being used or when the battery is low.



<https://support.ablenetinc.com/sh-qtf-quickstart>

## Breaks Happen - Warranty

Your loved one's voice is protected with an unconditional 5-year warranty. You can file anytime and we'll send a replacement right away.



<https://support.ablenetinc.com/ws-request>



## Your Speech App

Visit our Support Hub for more information on how to get started with the speech app chosen for your loved one.



<https://support.ablenetinc.com/shspeechapps>



## Resources

We've compiled several resources to help you and your loved one get started. Everything from incorporating communication into daily routines to modeling to motivating your loved one to communicate is available online.



<https://support.ablenetinc.com/sh-pcg-resources>



## Support

Funding Support  
(651) 294-2211  
[fundingservices@ablenetinc.com](mailto:fundingservices@ablenetinc.com)

ableCARE Product Support  
(651) 294-3101  
[ablecare@ablenetinc.com](mailto:ablecare@ablenetinc.com)

Support Hub  
<https://support.ablenetinc.com/quickstart>

# Let's go!



## Join ableFamilies!

Connect with other caregivers of QuickTalker Freestyle users for real ideas, real support, and real success!