Instructions For Use

Pillow Switch

Issue Date: February 26, 2024

Revision Table:

Revision	Date	Description
1.0	02/26/2024	Issue Date
1.1	05/12/2025	Revision of Reusable section
1.2	08/13/2026	Addition of sharp point risk statement

Model: 58750

UDID: 00186648000180

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
www.ablenetinc.com

EUCEREP
Roald Dahllaan 33, 5629MC
Eindhoven
The Netherlands
eucerep@eucerep.com

UK Responsible Person

EUCEREP LTD
7 Tibbets Close, Meeting Lane
Alcester, Warwickshire
B49 5QU
England, United Kingdom

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland
Telephone: +41 41 562 01 42

INTENDED PURPOSE

This switch is used to give access to items that are not normally accessible because of issues with cognitive or motor control.

CLINICAL BENEFITS OF THE DEVICE

Allows patients to access technology when they would not have the motor or cognitive skills to operate another device.

PERFORMANCE CHARACTERISTICS AND INFORMATION FOR HEALTH CARE PROFESSIONAL

Switch Type: Mechanical

Connection Type: Wired

Cord Length: 5 ft (152 cm)

Activation Type: Pressure

Activation Surface: 3.3-in/8-cm diameter

Activating Force: 6-oz/180-g

Travel Before Activation: 0.1-in/0.2-cm

Feedback: Auditory click, Tactile

Plug Size: 1/8-in/3.5-mm mono

Mounting Plate: Included Velcro and Safety Pin

Product Dimensions: 3.35-in L x 3.35-in W x 0.95-in H (8.6-cm L x 8.6-cm W x 2.5-cm H)

Circuit: Normally Open

Material: Housing - ABS Plastic

Suggested Operating Environment:

Ambient Temperature Range: 32°F - 95°F (0°C - 35°C)

Humidity: 10% - 95% (no condensation on device)

Suggested Transportation and Storage:

Ambient Temperature Range: -25°F - 158°F (-31°C - 70°C)

Humidity: 10% - 95% (no condensation on the device)

RISKS, WARNINGS, AND CONTRA-INDICATIONS

Pillow Switch is neither waterproof nor completely water resistant. Do not immerse Pillow Switch in water or any other liquid. Make sure that no liquids are spilled on Pillow Switch.


Pillow Switch is not recommended to be used as access to a call system.

Pillow Switch contains a functional safety pin with a sharp point. Risk of puncture injury. Adult supervision required when used by children. Inspect before each use and discontinue use if the pin becomes loose or damaged.

DEVICE OVERVIEW

1	Switch Top
2	1/8-in/3.5-mm mono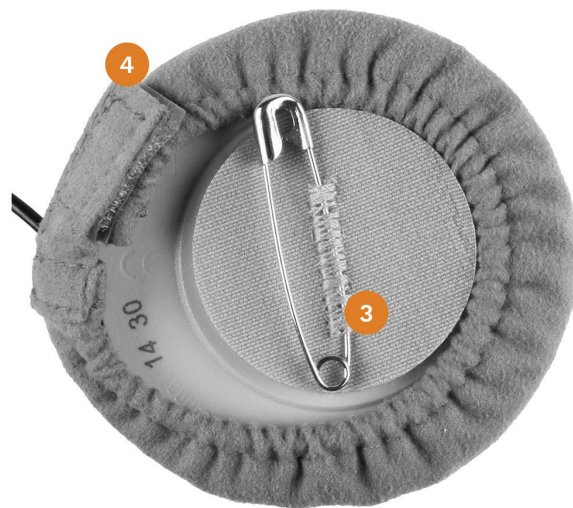

3	Safety pin and Velcro
4	Pillow Switch Cover Velcro clasp

USE INSTRUCTIONS

First Time Use

1. Plug Pillow Switch into any switch adapted device with a standard, mono 3.5mm switch jack.
2. Once the activation area is pressed, any attached device will activate.

Mounting

Pillow Switch easily fastens to a pillow or wheelchair cushion with safety pin and Velcro.

PREPARATORY TREATMENT

Cleaning and disinfecting your AbleNet assistive technology devices between uses is essential to the health and safety of the user. To help fight against the spread of germs, here are some helpful steps to cleaning and disinfecting your AbleNet device.

Always be sure to wear the appropriate personal protective equipment such as protective gloves and safety glasses.

Cleaning Your Device

Before disinfecting your device, remove any dirt or debris that may be on the device.

1. Turn the device off and remove the batteries.
2. Remove any large chunks, visible heavy dirt, or debris that is attached to the device.
3. Wipe down the device with cleaning wipes.
4. Use a cotton swab soaked with cleaner to get into any tight areas that were not accessible with the cleaning wipes.
5. Let the device completely dry.

Disinfect Your Device

Now that the device is clean, it is time to disinfect it. Changing protective gloves may be required before disinfecting the device to ensure no dirt or debris is transferred back to the device.

1. Make sure the device is still turned off and that the batteries are removed.
2. Use a clean towel or wipes with Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, or Fungicidal to disinfect the surfaces of the device. It is important to note that these chemicals only disinfect when they are wet and should be left wet on the device for at least 4-minutes.
3. Use a cotton swab soaked with Virucidal, Bactericidal, Pseudomonacidal, Tuberculocidal, or Fungicidal to disinfect the tight areas that were not accessible with the towel or wipes. Make sure the chemical stays wet on the device for at least 4-minutes.
4. Let the device air-dry entirely before using it.

TROUBLESHOOTING

Switch is Non-Responsive

1. Inspect cord for any physical damage.
2. Plug switch into another device that is working with another switch.

PREVENTATIVE AND REGULAR MAINTENANCE

Clean the Pillow Switch if necessary. No scheduled maintenance is required.

REUSABLE

If the device is intended to be reused, it must be cleaned and disinfected between uses in accordance with standard infection control procedures. Reuse is appropriate only if the device remains in good working condition and shows no signs of material degradation.

Cleaning and Disinfection Prior to Reuse

Refer to the PREPARATORY TREATMENT section, above.

Inspection Prior to Reuse

Before each reuse, inspect the device for:

- Physical damage or wear
- Signs of degradation to wiring, battery compartment, or connectors
- Corrosion, cracking, or discoloration of components

Discontinuation of Use

Discontinue use and dispose of the device according to local regulations if:

- Any signs of material degradation are present
- The device fails to operate reliably

Where appropriate, devices may be reused if they are reconditioned by the manufacturer or an authorized representative, in accordance with applicable safety and performance requirements.

DEVICES USED TOGETHER WITH

Switch Adapted Toys

HITCH 2

Blue 2 FT

Powerlink 4

Other adapters or toys that are designed with 3.5mm-mono plug for switch use.

PROPER DISPOSAL

Pillow Switch contains electronic components and should not be disposed of in your normal home or business waste collection. Pillow Switch should be disposed of with a licensed electronics recycler.

WHEN TO CONTACT A HEALTHCARE PROFESSIONAL

If you have difficulty accessing the switches or need assistance with product placement.

Change in your condition.

NOTICE OF SERIOUS INCIDENT

Notice - If any serious incident has occurred while using this device, seek medical attention and then - please report to AbleNet, Inc and your competent authority of the member state in which you are in.

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EUCEREP LTD
7 Tibbets Close, Meeting Lane
Alcester, Warwickshire
B49 5QU
Angleterre, Royaume-Uni

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