


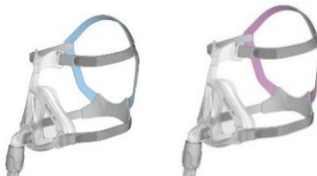



Crosswalk for ResMed full-face masks with magnetic clips¹


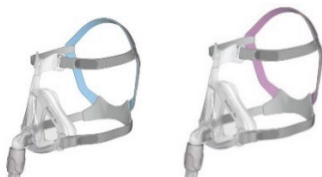

The following ResMed full-face masks – AirFit™ F20, AirFit F20 for Her, AirTouch™ F20 AirFit F30 and AirFit F30i – contain magnetic clips. Following the update of the existing contraindications and warnings for these masks (please refer to the Field Safety Notice we sent you), this document lists a number of alternative products from the ResMed portfolio to help you advise contraindicated patients to a mask without magnetic clips.

For compatibility information, see the product user /clinical guide or the mask/device compatibility list at [ResMed.com/downloads/masks](https://www.resmed.com/downloads/masks).


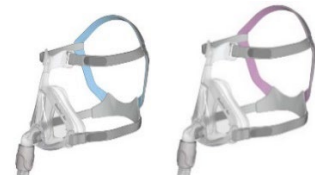

This information serves as a guidance for you to better identify the recommended mask replacement for your patient. To ensure the best fit for your patient, please perform a fitting test and refer to the relevant user /clinical guide for more detailed instructions, as well as precautions.

	Full-face mask with magnetic clips ¹ :	Alternative full-face masks without magnetic clips:	
	AirFit™ F20/ AirFit F20 for Her	Quattro™ Air/ Quattro Air for Her	Mirage Quattro™
			
Pressure range	3-40 cmH ₂ O	3-40 cmH ₂ O	4-40 cmH ₂ O
Sizing conversions*	AirFit F20 SML	Quattro Air SML	Mirage Quattro SML
	AirFit F20 MED	Quattro Air MED	Mirage Quattro MED
	AirFit F20 LGE	Quattro Air LGE	Mirage Quattro LGE
	AirFit F20 for Her SML	Quattro Air XSML/Quattro Air SML for Her	Mirage Quattro XSML / Mirage Quattro SML
	AirFit F20 for Her MED	Quattro Air MED for Her	Mirage Quattro MED
Mask systems product codes	63405-AirFit F20 SML Sys - EU1 63406-AirFit F20 MED Sys - EU1 63407-AirFit F20 LGE Sys - EU1		61205-Mirage Quattro FFM SML- EUR1 61206-Mirage Quattro FFM MED- EUR1 61207-Mirage Quattro FFM LGE- EUR1




Note: Other types of masks without magnetic clips include Mirage™ FX, AirFit N20 Classic, AirFit N30i, AirFit N30 (nasal masks) and AirFit P10, AirFit P30i, Swift™ FX (nasal pillows masks). Please visit [ResMed.com](https://www.resmed.com) for more information regarding these masks.

	Full-face mask with magnetic clips ¹ :	Alternative full-face masks without magnetic clips:	
	AirTouch™ F20	Quattro™ Air/ Quattro Air for Her	Mirage Quattro™
			
Pressure range	3-40 cmH ₂ O	3-40 cmH ₂ O	4-40 cmH ₂ O
Sizing conversions*	AirTouch F20 SML	Quattro Air SML	Mirage Quattro SML
	AirTouch F20 MED	Quattro Air MED	Mirage Quattro MED
	AirTouch F20 LGE	Quattro Air LGE	Mirage Quattro LGE
Mask systems product codes			

Note: Other types of masks without magnetic clips include Mirage FX, AirFit N20 Classic, AirFit N30i, AirFit N30 (nasal masks) and AirFit P10, AirFit P30i, Swift FX (nasal pillows masks). Please visit [ResMed.com](https://www.ResMed.com) for more information regarding these masks.

	Full-face mask with magnetic clips ¹ :	Alternative full-face masks without magnetic clips:	
	AirFit™ F30	Quattro™ Air/ Quattro Air for Her	Mirage Quattro™
			
Pressure range	3-30 cmH ₂ O	3-40 cmH ₂ O	4-40 cmH ₂ O
Sizing conversions*	AirFit F30 SML	Quattro Air SML	Mirage Quattro SML
	AirFit F30 MED	Quattro Air MED/ Quattro Air LGE	Mirage Quattro MED/ Mirage Quattro LGE
Mask systems product codes	64110-AirFit F30 SML Sys - EU 64111-AirFit F30 MED Sys – EU		

Note: Other types of masks without magnetic clips include Mirage FX, AirFit N20 Classic, AirFit N30i, AirFit N30 (nasal masks) and AirFit P10, AirFit P30i, Swift FX (nasal pillows masks). Please visit [ResMed.com](https://www.resmed.com) for more information regarding these masks.

	Full-face mask with magnetic clips ¹ :	Alternative full-face masks without magnetic clips:	
	AirFit™ F30i	Quattro™ Air/ Quattro Air for Her	Mirage Quattro™
			
Pressure range	4-30 cmH ₂ O	3-40 cmH ₂ O	4-40 cmH ₂ O
Sizing conversions*	AirFit F30i S/SML	Quattro Air XSML/ Quattro Air SML for Her	Mirage Quattro XSML/ Mirage Quattro SML
	AirFit F30i S/STD	Quattro Air SML	Mirage Quattro SML
	AirFit F30i M/STD	Quattro Air MED	Mirage Quattro MED
	AirFit F30i M/LGE	Quattro Air MED	Mirage Quattro MED
	AirFit F30i W/STD	Quattro Air LGE	Mirage Quattro LGE
Mask systems product codes	63310-AirFit F30i SYS - S/SML - EU 63311-AirFit F30i SYS - S/STD - EU 63312-AirFit F30i SYS - M/STD - EU 63313-AirFit F30i SYS - W/STD - EU 63314-AirFit F30i SYS - M/LGE - EU		

Note: Other types of masks without magnetic clips include Mirage FX, AirFit N20 Classic, AirFit N30i, AirFit N30 (nasal masks) and AirFit P10, AirFit P30i, Swift FX (nasal pillows masks). Please visit [ResMed.com](https://www.ResMed.com) for more information regarding these masks.



This content is intended for health professionals only. Please refer to the clinical and user guides for relevant information related to any warnings and precautions to be considered before and during use of the product.

1. The mask contains magnets that may interfere with certain implants or medical devices. They are not to be used by patients where they, or anyone in close physical contact while using the mask, have active medical implants that interact with magnets and/or metallic implants/objects containing ferromagnetic material. Keep the mask magnets at a safe distance of at least 6 inches (150 mm) away from implants or medical devices that may be adversely affected by magnetic interference. Please refer to the user guide for complete information, including magnet contraindications and warning.

*Size availability may vary across regions.

AirFit, AirTouch, Mirage, Mirage Quattro, Quattro, Swift are trademarks of ResMed. Specifications may change without notice.
© 2023 ResMed. 10113749/4-LBN 2024-02








Crosswalk for ResMed nasal masks with magnetic clips¹






The following ResMed nasal masks – AirFit™ N20, AirFit N20 for Her, AirTouch™ N20, AirFit N10 and AirFit N10 for Her – contain magnetic clips. Following the update of the existing contraindications and warnings for these masks (please refer to the Field Safety Notice we sent you), this document lists a number of alternative products from the existing ResMed portfolio to help you advise contraindicated patients to a mask without magnetic clips.

For compatibility information, see the product user/clinical guide or the mask/device compatibility list at [ResMed.com/downloads/masks](https://www.resmed.com/downloads/masks).






This information serves as a guidance for you to better identify the recommended mask replacement for your patient. To ensure the best fit for your patient, please perform a fitting test and refer to the relevant user/clinical guide for more detailed instructions, as well as precautions.

	Nasal mask with magnetic clips ¹ :	Alternative nasal masks without magnetic clips:			
	AirFit™ N20/ AirFit N20 for Her	Mirage™ FX/ Mirage FX for Her	AirFit N20 Classic	AirFit N30i	AirFit N30
					
Pressure range	4-30 cmH ₂ O	4-20 cmH ₂ O	4-30 cmH ₂ O	4-25 cmH ₂ O	4-20 cmH ₂ O
Sizing conversions*	AirFit N20 SML	Mirage FX Standard/ Mirage FX for Her	AirFit N20 Classic SML	AirFit N30i S/SML	AirFit N30 ²
	AirFit N20 MED	Mirage FX Standard	AirFit N20 Classic MED	AirFit N30i M/STD	
	AirFit N20 LGE	Mirage FX Wide	AirFit N20 Classic LGE	AirFit N30i W/STD	
	AirFit N20 for Her SML	Mirage FX for Her	AirFit N20 Classic SML	AirFit N30i S/SML	
Mask systems product codes	63511-AirFit N20 MED Sys - EU1 63512-AirFit N20 LGE Sys - EU1			63817-AirFit N30i SYS S/SML Quiet EU 63818-AirFit N30i SYS SW/SML Quiet EU 63819-AirFit N30i SYS M/STD Quiet EU 63820-AirFit N30i SYS W/STD Quiet EU 63883-EU2-AirFit N30i SYS W/STD MultiH EU	64204-AirFit N30 Mask SYS EU

Note: Nasal pillows masks without magnetic clips include AirFit P10, AirFit P30i, Swift FX. Full-face masks without magnetic clips include Quattro Air and Mirage Quattro. Please visit [ResMed.com](https://www.ResMed.com) for more information regarding these masks.

	Nasal mask with magnetic clips ¹ :	Alternative nasal masks without magnetic clips:			
	AirTouch™ N20	Mirage™ FX/ Mirage FX for Her	AirFit N20 Classic	AirFit N30i	AirFit N30
					
Pressure range	4-30 cmH ₂ O	4-20 cmH ₂ O	4-30 cmH ₂ O	4-25 cmH ₂ O	4-20 cmH ₂ O
Sizing conversions*	AirTouch N20 SML	Mirage FX Standard / Mirage FX for Her	AirFit N20 Classic SML	AirFit N30i S/SML	AirFit N30 ²
	AirTouch N20 MED	Mirage FX Standard	AirFit N20 Classic MED	AirFit N30i M/STD	
	AirTouch N20 LGE	Mirage FX Wide	AirFit N20 Classic LGE	AirFit N30i W/STD	
Mask systems product codes				63817 -AirFit N30i SYS S/SML Quiet EU 63818 -AirFit N30i SYS SW/SML Quiet EU 63819 -AirFit N30i SYS M/STD Quiet EU 63820 -AirFit N30i SYS W/STD Quiet EU 63883 -EU2-AirFit N30i SYS W/STD MultiH EU	64204 -AirFit N30 Mask SYS EU

Note: Nasal pillows masks without magnetic clips include AirFit P10, AirFit P30i, Swift FX. Full-face masks without magnetic clips include Quattro Air and Mirage Quattro. Please visit [ResMed.com](https://www.ResMed.com) for more information regarding these masks.

	Nasal mask with magnetic clips ¹ :	Alternative nasal masks without magnetic clips:			
	AirFit™ N10/AirFit N10 for Her**	Mirage™ FX/Mirage FX for Her	AirFit N20 Classic	AirFit N30i	AirFit N30
					
Pressure range	4-30 cmH ₂ O	4-20 cmH ₂ O	4-30 cmH ₂ O	4-25 cmH ₂ O	4-20 cmH ₂ O
Sizing conversions*	AirFit N10 SML	Mirage FX Standard / Mirage FX for Her	AirFit N20 Classic SML	AirFit N30i S/SML	AirFit N30 ²
	AirFit N10 MED	Mirage FX Standard	AirFit N20 Classic MED	AirFit N30i M/STD	
	AirFit N10 WIDE	Mirage FX Wide	AirFit N20 Classic LGE	AirFit N30i W/STD	
	AirFit N10 for Her SML	Mirage FX for Her	AirFit N20 Classic SML	AirFit N30i S/SML	
Mask systems product codes					

Note: Nasal pillows masks without magnetic clips include AirFit P10, AirFit P30i, Swift FX. Full-face masks without magnetic clips include Quattro Air and Mirage Quattro. Please visit [ResMed.com](https://www.ResMed.com) for more information regarding these masks.



This content is intended for health professionals only. Please refer to the clinical and user guides for relevant information related to any warnings and precautions to be considered before and during use of the product.

1. The mask contains magnets that may interfere with certain implants or medical devices. They are not to be used by patients where they, or anyone in close physical contact while using the mask, have active medical implants that interact with magnets and/or metallic implants/objects containing ferromagnetic material. Keep the mask magnets at a safe distance of at least 6 inches (150 mm) away from implants or medical devices that may be adversely affected by magnetic interference. Please refer to the user guide for complete information, including magnet contraindications and warning.
2. AirFit N30 is supplied with both S and M cushion sizes.

*Size availability may vary across regions.

**AirFit N10 and AirFit N10 for Her have been phased out of ResMed's portfolio. However, as they may still be present on the market they are listed in this document to assist you in transitioning contraindicated patients who are still using these masks.

AirFit, AirTouch, Mirage, Swift are trademarks of ResMed. Specifications may change without notice.

© 2023 ResMed. 10113747/3-LBN 2023-10