








## Crosswalk for ResMed full-face masks with magnetic clips<sup>1</sup>






The following ResMed full-face masks – AirFit™ F20, AirFit F20 for Her, AirTouch™ F20 AirFit F30 and AirFit F30i – contain magnetic clips. Following the update of the existing contraindications and warnings for these masks (please refer to the Field Safety Notice we sent you), this document lists a number of alternative products from the ResMed portfolio to help you advise contraindicated patients to a mask without magnetic clips.

For compatibility information, see the product user/clinical guide or the mask/device compatibility list at [ResMed.com/downloads/masks](https://www.resmed.com/downloads/masks).




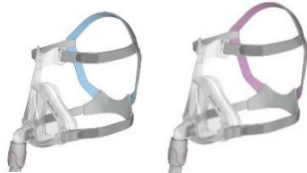

This information serves as a guidance for you to better identify the recommended mask replacement for your patient. To ensure the best fit for your patient, please perform a fitting test and refer to the relevant user/clinical guide for more detailed instructions, as well as precautions.

	Full-face mask with magnetic clips <sup>1</sup> :	Alternative full-face masks without magnetic clips:			
	AirFit™ F20 /AirFit F20 for Her	AirFit F20 Non Magnetic	AirFit F30i Non Magnetic	Quattro™ Air/ Quattro Air for Her	Mirage Quattro™
					
<b>Pressure range</b>	3-40 cmH <sub>2</sub> O	3-40 cmH <sub>2</sub> O	4-30 cmH <sub>2</sub> O	3-40 cmH <sub>2</sub> O	4-40 cmH <sub>2</sub> O
<b>Sizing conversions*</b>	AirFit F20 SML	AirFit F20 Non Magnetic SML	AirFit F30i Non Magnetic S/STD	Quattro Air SML	Mirage Quattro SML
	AirFit F20 MED	AirFit F20 Non Magnetic MED	AirFit F30i Non Magnetic M/STD	Quattro Air MED	Mirage Quattro MED
	AirFit F20 LGE	AirFit F20 Non Magnetic LGE	AirFit F30i Non Magnetic W/STD	Quattro Air LGE	Mirage Quattro LGE
	AirFit F20 for Her SML	AirFit F20 Non Magnetic SML	AirFit F30i Non Magnetic S/SML	Quattro Air XSML/Quattro Air SML for Her	Mirage Quattro XSML / Mirage Quattro SML
	AirFit F20 for Her MED	AirFit F20 Non Magnetic MED	AirFit F30i Non Magnetic M/STD	Quattro Air MED for Her	Mirage Quattro MED
<b>Mask systems product codes</b>	<b>63405</b> -AirFit F20 SML Sys - EU1 <b>63406</b> -AirFit F20 MED Sys - EU1 <b>63407</b> -AirFit F20 LGE Sys - EU1 <b>63408</b> -AirFit F20 for Her SML - EU1 <b>63409</b> -AirFit F20 for Her MED - EU1 <b>64005</b> -AirFit F20 SML Sys Quiet-EU1 <b>64006</b> -AirFit F20 MED Sys Quiet-EU1 <b>64007</b> -AirFit F20 LGE Sys Quiet-EU1 <b>64008</b> -AirFit F20 for Her SML Quiet-EU1 <b>64009</b> -AirFit F20 for Her MED Quiet-EU1	<b>64031</b> – AirFit F20 NON MAGN SML QUIET- EU <b>64032</b> AirFit F20 NON MAGN MED QUIET- EU <b>64033</b> - AirFit F20 NON MAGN LGE QUIET- EU	<b>63341</b> - AirFit F30i Non Magnetic S/SML  <b>63343</b> - Non Magnetic M/STD AirFit F30i  <b>63344</b> - AirFit F30i Non Magnetic W/STD	<b>62704</b> -Quattro Air FFM XSML-EUR1 <b>62705</b> -Quattro Air FFM SML-EUR1 <b>62706</b> -Quattro Air FFM MED-EUR1 <b>62707</b> -Quattro Air FFM LGE-EUR1	<b>61204</b> -Mirage Quattro FFM XSML-EUR1 <b>61205</b> -Mirage Quattro FFM SML-EUR1 <b>61206</b> -Mirage Quattro FFM MED-EUR1 <b>61207</b> -Mirage Quattro FFM LGE-EUR1




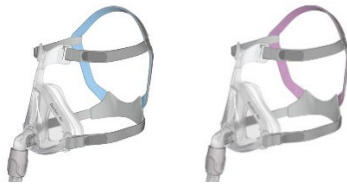

Note: Other types of masks without magnetic clips include Mirage™ FX, AirFit N20 Classic, AirFit N30i, AirFit N30 (nasal masks) and AirFit P10, AirFit P30i, Swift™ FX (nasal pillows masks). Please visit ResMed.com for more information regarding these masks.

	Full-face mask with magnetic clips <sup>1</sup> :	Alternative full-face masks without magnetic clips:			
	AirTouch™ F20	AirFit F20 Non Magnetic	AirFit F30i Non Magnetic	Quattro™ Air/Quattro Air for Her	Mirage Quattro™
					
<b>Pressure range</b>	3-40 cmH <sub>2</sub> O	3-40 cmH <sub>2</sub> O	4-30 cmH <sub>2</sub> O	3-40 cmH <sub>2</sub> O	4-40 cmH <sub>2</sub> O
<b>Sizing conversions *</b>	AirTouch F20 SML	AirFit F20 Non Magnetic SML	AirFit F30i Non Magnetic S/STD	Quattro Air SML	Mirage Quattro SML
	AirTouch F20 MED	AirFit F20 Non Magnetic MED	AirFit F30i Non Magnetic M/STD	Quattro Air MED	Mirage Quattro MED
	AirTouch F20 LGE	AirFit F20 Non Magnetic LGE	AirFit F30i Non Magnetic W/STD	Quattro Air LGE	Mirage Quattro LGE
<b>Mask systems product codes</b>	<b>63034</b> - AirTouch F20 SML Sys - EU <b>63035</b> - AirTouch F20 MED Sys EU <b>63036</b> - AirTouch F20 LGE Sys EU	<b>64031</b> – AirFit F20 NON MAGN SML QUIET- EU <b>64032</b> AirFit F20 NON MAGN MED QUIET- EU <b>64033</b> - AirFit F20 NON MAGN LGE QUIET- EU	<b>63341</b> - AirFit F30i Non Magnetic S/SML  <b>63343</b> - Non Magnetic M/STD AirFit F30i  <b>63344</b> - AirFit F30i Non Magnetic W/STD	<b>62704</b> -Quattro Air FFM XSML-EUR1 <b>62705</b> -Quattro Air FFM SML-EUR1 <b>62706</b> -Quattro Air FFM MED-EUR1 <b>62707</b> -Quattro Air FFM LGE-EUR1	<b>61204</b> -Mirage Quattro FFM XSML-EUR1 <b>61205</b> -Mirage Quattro FFM SML-EUR1 <b>61206</b> -Mirage Quattro FFM MED-EUR1 <b>61207</b> -Mirage Quattro FFM LGE-EUR1

Note: Other types of masks without magnetic clips include Mirage FX, AirFit N20 Classic, AirFit N30i, AirFit N30 (nasal masks) and AirFit P10, AirFit P30i, Swift FX (nasal pillows masks). Please visit [ResMed.com](http://ResMed.com) for more information regarding these masks.

	Full-face mask with magnetic clips <sup>1</sup> :	Alternative full-face masks without magnetic clips:			
	AirFit™ F30	AirFit F30i Non Magnetic	AirFit F20 Non Magnetic	Quattro™ Air/ Quattro Air for Her	Mirage Quattro™
					
<b>Pressure range</b>	3-30 cmH <sub>2</sub> O	4-30 cmH <sub>2</sub> O	3-40 cmH <sub>2</sub> O	3-40 cmH <sub>2</sub> O	4-40 cmH <sub>2</sub> O
<b>Sizing conversions *</b>	AirFit F30 SML	AirFit F30i Non Magnetic S/STD	AirFit F20 Non Magnetic SML	Quattro Air SML	Mirage Quattro SML
	AirFit F30 MED	AirFit F30i Non Magnetic M/STD	AirFit F20 Non Magnetic MED/ AirFit F20 Non Magnetic LGE	Quattro Air MED/ Quattro Air LGE	Mirage Quattro MED/ Mirage Quattro LGE
<b>Mask systems product codes</b>	64110-AirFit F30 SML Sys - EU 64111-AirFit F30 MED Sys – EU	63341 - AirFit F30i Non Magnetic S/SML  63343 - Non Magnetic M/STD AirFit F30i  63344 - AirFit F30i Non Magnetic W/STD	64031 – AirFit F20 NON MAGN SML QUIET- EU 64032 AirFit F20 NON MAGN MED QUIET- EU 64033 - AirFit F20 NON MAGN LGE QUIET- EU	62704-Quattro Air FFM XSML-EUR1 62705-Quattro Air FFM SML-EUR1 62706-Quattro Air FFM MED-EUR1 62707-Quattro Air FFM LGE-EUR1	61204-Mirage Quattro FFM XSML-EUR1 61205-Mirage Quattro FFM SML-EUR1 61206-Mirage Quattro FFM MED-EUR1 61207-Mirage Quattro FFM LGE-EUR1

Note: Other types of masks without magnetic clips include Mirage FX, AirFit N20 Classic, AirFit N30i, AirFit N30 (nasal masks) and AirFit P10, AirFit P30i, Swift FX (nasal pillows masks). Please visit [ResMed.com](http://ResMed.com) for more information regarding these masks.

	Full-face mask with magnetic clips <sup>1</sup> :	Alternative full-face masks without magnetic clips:			
	AirFit™ F30i	AirFit F30i Non Magnetic	AirFit F20 Non Magnetic	Quattro™ Air/ Quattro Air for Her	Mirage Quattro™
					
<b>Pressure range</b>	4-30 cmH <sub>2</sub> O	4-30 cmH <sub>2</sub> O	3-40 cmH <sub>2</sub> O	3-40 cmH <sub>2</sub> O	4-40 cmH <sub>2</sub> O
<b>Sizing conversions*</b>	AirFit F30i S/SML	AirFit F30i Non Magnetic S/SML	AirFit F20 Non Magnetic SML	Quattro Air XSML/ Quattro Air SML for Her	Mirage Quattro XSML/ Mirage Quattro SML
	AirFit F30i S/STD	AirFit F30i Non Magnetic S/STD	AirFit F20 Non Magnetic SML	Quattro Air SML	Mirage Quattro SML
	AirFit F30i M/STD	AirFit F30i Non Magnetic M/STD	AirFit F20 Non Magnetic MED	Quattro Air MED	Mirage Quattro MED
	AirFit F30i M/LGE	AirFit F30i Non Magnetic M/LGE	AirFit F20 Non Magnetic MED	Quattro Air MED	Mirage Quattro MED
	AirFit F30i W/STD	AirFit F30i Non Magnetic W/STD	AirFit F20 Non Magnetic LGE	Quattro Air LGE	Mirage Quattro LGE
<b>Mask systems product codes</b>	<b>63310</b> -AirFit F30i SYS - S/SML - EU <b>63312</b> -AirFit F30i SYS - M/STD - EU <b>63313</b> -AirFit F30i SYS - W/STD - EU <b>63314</b> -AirFit F30i SYS - M/LGE - EU	<b>63341</b> - AirFit F30i Non Magnetic S/SML  <b>63343</b> - Non Magnetic M/STD AirFit F30i  <b>63344</b> - AirFit F30i Non Magnetic W/STD	<b>64031</b> – AirFit F20 NON MAGN SML QUIET- EU <b>64032</b> AirFit F20 NON MAGN MED QUIET- EU <b>64033</b> - AirFit F20 NON MAGN LGE QUIET- EU	<b>62704</b> -Quattro Air FFM XSML-EUR1 <b>62705</b> -Quattro Air FFM SML-EUR1 <b>62706</b> -Quattro Air FFM MED-EUR1 <b>62707</b> -Quattro Air FFM LGE-EUR1	<b>61204</b> -Mirage Quattro FFM XSML-EUR1 <b>61205</b> -Mirage Quattro FFM SML-EUR1 <b>61206</b> -Mirage Quattro FFM MED-EUR1 <b>61207</b> -Mirage Quattro FFM LGE-EUR1

Note: Other types of masks without magnetic clips include Mirage FX, AirFit N20 Classic, AirFit N30i, AirFit N30 (nasal masks) and AirFit P10, AirFit P30i, Swift FX (nasal pillows masks). Please visit [ResMed.com](http://ResMed.com) for more information regarding these masks.



This content is intended for health professionals only. Please refer to the clinical and user guides for relevant information related to any warnings and precautions to be considered before and during use of the product.

1. The mask contains magnets that may interfere with certain implants or medical devices. They are not to be used by patients where they, or anyone in close physical contact while using the mask, have active medical implants that interact with magnets and/or metallic implants/objects containing ferromagnetic material. Keep the mask magnets at a safe distance of at least 6 inches (150 mm) away from implants or medical devices that may be adversely affected by magnetic interference. Please refer to the user guide for complete information, including magnet contraindications and warning.

\*Size availability may vary across regions.

AirFit, AirTouch, Mirage, Mirage Quattro, Quattro, Swift are trademarks of ResMed. Specifications may change without notice.  
© 2023 ResMed. 10113748/4-IRN 2024-02








## Crosswalk for ResMed nasal masks with magnetic clips<sup>1</sup>

The following ResMed nasal masks – AirFit™ N20, AirFit N20 for Her, AirTouch™ N20, AirFit N10 and AirFit N10 for Her – contain magnetic clips. Following the update of the existing contraindications and warnings for these masks (please refer to the Field Safety Notice we sent you), this document lists a number of alternative products from the existing ResMed portfolio to help you advise contraindicated patients to a mask without magnetic clips.






For compatibility information, see the product user/clinical guide or the mask/device compatibility list at [ResMed.com/downloads/masks](https://www.resmed.com/downloads/masks).

This information serves as a guidance for you to better identify the recommended mask replacement for your patient. To ensure the best fit for your patient, please perform a fitting test and refer to the relevant user/clinical guide for more detailed instructions, as well as precautions.






	Nasal mask with magnetic clips <sup>1</sup> :	Alternative nasal masks without magnetic clips:			
	AirFit™ N20/ AirFit N20 for Her	Mirage™ FX/ Mirage FX for Her	AirFit N20 Classic	AirFit N30i	AirFit N30
					
<b>Pressure range</b>	4-30 cmH <sub>2</sub> O	4-20 cmH <sub>2</sub> O	4-30 cmH <sub>2</sub> O	4-25 cmH <sub>2</sub> O	4-20 cmH <sub>2</sub> O
<b>Sizing conversions*</b>	AirFit N20 SML	Mirage FX Standard/ Mirage FX for Her	AirFit N20 Classic SML	AirFit N30i S/SML	AirFit N30 <sup>2</sup>
	AirFit N20 MED	Mirage FX Standard	AirFit N20 Classic MED	AirFit N30i M/STD	
	AirFit N20 LGE	Mirage FX Wide	AirFit N20 Classic LGE	AirFit N30i W/STD	
	AirFit N20 for Her SML	Mirage FX for Her	AirFit N20 Classic SML	AirFit N30i S/SML	
<b>Mask systems product codes</b>					

*Note: Nasal pillows masks without magnetic clips include AirFit P10, AirFit P30i, Swift FX. Full-face masks without magnetic clips include Quattro Air and Mirage Quattro. Please visit ResMed.com for more information regarding these masks.*



	Nasal mask with magnetic clips <sup>1</sup> :	Alternative nasal masks without magnetic clips:			
	AirTouch™ N20	Mirage™ FX/ Mirage FX for Her	AirFit N20 Classic	AirFit N30i	AirFit N30
					
<b>Pressure range</b>	4-30 cmH <sub>2</sub> O	4-20 cmH <sub>2</sub> O	4-30 cmH <sub>2</sub> O	4-25 cmH <sub>2</sub> O	4-20 cmH <sub>2</sub> O
<b>Sizing conversions*</b>	AirTouch N20 SML	Mirage FX Standard / Mirage FX for Her	AirFit N20 Classic SML	AirFit N30i S/SML	AirFit N30 <sup>2</sup>
	AirTouch N20 MED	Mirage FX Standard	AirFit N20 Classic MED	AirFit N30i M/STD	
	AirTouch N20 LGE	Mirage FX Wide	AirFit N20 Classic LGE	AirFit N30i W/STD	
<b>Mask systems product codes</b>					

*Note: Nasal pillows masks without magnetic clips include AirFit P10, AirFit P30i, Swift FX. Full-face masks without magnetic clips include Quattro Air and Mirage Quattro. Please visit [ResMed.com](https://www.ResMed.com) for more information regarding these masks.*

	Nasal mask with magnetic clips <sup>1</sup> :	Alternative nasal masks without magnetic clips:			
	AirFit™ N10/AirFit N10 for Her**	Mirage™ FX/Mirage FX for Her	AirFit N20 Classic	AirFit N30i	AirFit N30
					
<b>Pressure range</b>	4-30 cmH <sub>2</sub> O	4-20 cmH <sub>2</sub> O	4-30 cmH <sub>2</sub> O	4-25 cmH <sub>2</sub> O	4-20 cmH <sub>2</sub> O
<b>Sizing conversions*</b>	AirFit N10 SML	Mirage FX Standard / Mirage FX for Her	AirFit N20 Classic SML	AirFit N30i S/SML	AirFit N30 <sup>2</sup>
	AirFit N10 MED	Mirage FX Standard	AirFit N20 Classic MED	AirFit N30i M/STD	
	AirFit N10 WIDE	Mirage FX Wide	AirFit N20 Classic LGE	AirFit N30i W/STD	
	AirFit N10 for Her SML	Mirage FX for Her	AirFit N20 Classic SML	AirFit N30i S/SML	
<b>Mask systems product codes</b>					

Note: Nasal pillows masks without magnetic clips include AirFit P10, AirFit P30i, Swift FX. Full-face masks without magnetic clips include Quattro Air and Mirage Quattro. Please visit ResMed.com for more information regarding these masks.



This content is intended for health professionals only. Please refer to the clinical and user guides for relevant information related to any warnings and precautions to be considered before and during use of the product.

1. The mask contains magnets that may interfere with certain implants or medical devices. They are not to be used by patients where they, or anyone in close physical contact while using the mask, have active medical implants that interact with magnets and/or metallic implants/objects containing ferromagnetic material. Keep the mask magnets at a safe distance of at least 6 inches (150 mm) away from implants or medical devices that may be adversely affected by magnetic interference. Please refer to the user guide for complete information, including magnet contraindications and warning.
2. AirFit N30 is supplied with both S and M cushion sizes.

\*Size availability may vary across regions.

\*\*AirFit N10 and AirFit N10 for Her have been phased out of ResMed's portfolio. However, as they may still be present on the market they are listed in this document to assist you in transitioning contraindicated patients who are still using these masks.

AirFit, AirTouch, Mirage, Swift are trademarks of ResMed. Specifications may change without notice.

© 2023 ResMed. 10113747/3-IRN 2023-10





## Crosswalk for ResMed non-vented masks with magnetic clips<sup>1</sup>

The AirFit™ F20 NV ResMed full-face non-vented mask contains magnetic clips. Following the update of the existing contraindications and warnings for this mask (please refer to the Field Safety Notice we sent you), this document lists an alternative product from the existing ResMed portfolio to help you advise contraindicated patients to a mask without magnetic clips.

For compatibility information, see the product user/clinical guide or the mask/device compatibility list at [ResMed.com/downloads/masks](https://www.resmed.com/downloads/masks).

This information serves as a guidance for you to better identify the recommended mask replacement for your patient. To ensure the best fit for your patient, please perform a fitting test and refer to the relevant user/clinical guide for more detailed instructions, as well as precautions.



	Full-face non-vented mask with magnetic clips <sup>1</sup> :	Alternative full-face non-vented mask without magnetic clips:
	AirFit™ F20 NV	Quattro™ Air NV
		
<b>Pressure range</b>	3-40 cmH <sub>2</sub> O	0-40 cmH <sub>2</sub> O
<b>Sizing conversions*</b>	AirFit F20 NV SML	NV Quattro Air FFM SML
	AirFit F20 NV MED	NV Quattro Air FFM MED
	AirFit F20 NV LGE	NV Quattro Air FFM LGE
<b>Mask systems product codes</b>		

This content is intended for health professionals only. Please refer to the clinical and user guides for relevant information related to any warnings and precautions to be considered before and during use of the product.

1. The mask contains magnets that may interfere with certain implants or medical devices. They are not to be used by patients where they, or anyone in close physical contact while using the mask, have active medical implants that interact with magnets and/or metallic implants/objects containing ferromagnetic material. Keep the mask magnets at a safe distance of at least 6 inches (150 mm) away from implants or medical devices that may be adversely affected by magnetic interference. Please refer to the user guide for complete information, including magnet contraindications and warning.

\*Size availability may vary across regions.