

For people living with PAH*

Start strong with RemunityPRO™

Your partner in Remodulin therapy, the RemunityPRO pump was thoughtfully designed with patient feedback



Starting pump therapy



Educational resources



Access & support



*Pulmonary arterial hypertension.

IMPORTANT SAFETY INFORMATION FOR REMUNITYPRO

Indication

The RemunityPRO Pump for Remodulin (treprostinil) Injection is intended for continuous subcutaneous delivery of Remodulin (treprostinil) Injection for use in patients, ages 17 and older.

Warnings and Cautions

Do not use the system outside the conditions listed in the User Guide. Retain the User Guide for future reference. Refer to the User Guide for all warnings and cautions. Failure to comply with the following warnings and cautions may result in harm.

IMPORTANT SAFETY INFORMATION FOR REMODULIN

What is Remodulin?

Remodulin is a prescription medication used to treat adults with pulmonary arterial hypertension (PAH; WHO Group 1), which is high blood pressure in the arteries of your lungs. Remodulin can reduce symptoms associated with exercise. Remodulin was studied mainly in patients with NYHA Functional Class II-IV symptoms. It is not known if Remodulin is safe and effective in children.

In people with PAH who need to switch from epoprostenol, Remodulin is approved to slow the worsening of symptoms.

Before you take Remodulin, tell your healthcare provider if you:

- Have other medical conditions or take other medicines that may affect your use of Remodulin by increasing the risk of side effects or decreasing the drug's effectiveness.

Please see Important Safety Information for RemunityPRO and Remodulin on page 8-9 and the Full Prescribing Information and Patient Information for Remodulin in pocket.



The RemunityPRO™ pump delivers Remodulin® 24/7, so you always know you're getting the treatment you need

Is subcutaneous (SC) delivery with Remodulin the right therapy for your PAH?

If you and your doctor aren't seeing the improvements you had hoped for in your exercise ability and PAH symptoms, it may be time to consider a prostacyclin treatment like Remodulin.



As the #1 most prescribed pump therapy for PAH, Remodulin has been trusted by patients and their doctors for more than 20 years.

In as little as 3 months, Remodulin may:



Help increase your exercise capacity



Lessen PAH symptoms (such as shortness of breath and fatigue)



Help you get back to doing more of the things you love

IMPORTANT SAFETY INFORMATION FOR REMUNITYPRO (CONT'D)

Warnings and Cautions (Cont'd)

Limited to use with Remodulin. Only RemunityPRO cassettes may be used with the RemunityPRO pump.

IMPORTANT SAFETY INFORMATION FOR REMODULIN (CONT'D)

Before you take Remodulin, tell your healthcare provider if you: (Cont'd)

- Have liver problems. Your Remodulin dose may need to be adjusted if you have liver problems.

Please see Important Safety Information for RemunityPRO and Remodulin on page 8-9 and the Full Prescribing Information and Patient Information for Remodulin in pocket.

RemunityPRO™ was optimized with patient feedback to better fit your routine

Starting pump therapy may feel like a lot, but it's a positive step toward doing more of the things you love. You're never on your own—one-on-one education and support will bring you up to speed and keep you going every step of the way.



RemunityPRO is a pump system similar to those used by people with diabetes. It was designed specifically for PAH, and improved from the first generation of Remunity to optimize the day-to-day pump experience.

“

I am feeling more confident that I can find a pump that might be better for me, and that takes away some of my anxiety.

—Remodulin patient



IMPORTANT SAFETY INFORMATION FOR REMUNITYPRO (CONT'D)

Warnings and Cautions (Cont'd)

RemunityPRO pump is for use only with FDA-cleared infusion sets: Medtronic Quick-set Infusion Set (MMT-392, MMT-393), Neria Guard (704060-5229, 704060-5226), Medtronic Silhouette Infusion Set (MMT-373), and Smiths Medical Cleo 90 Infusion Set (21-7230-24, 21-7220-24).

IMPORTANT SAFETY INFORMATION FOR REMODULIN (CONT'D)

Before you take Remodulin, tell your healthcare provider if you: (Cont'd)

- Have low blood pressure or bleeding problems.



Meet your RemunityPRO™

Innovation that delivers

RemunityPRO is a small, user-friendly pump thoughtfully designed for people living with PAH. It delivers advanced technology and features that help make starting and staying on Remodulin less complicated.

RemunityPRO fits into your personal routine, with:

- A small, discreet pump that fits in a pocket or purse
- Continuous delivery of Remodulin under the skin

Optimized features include:

- A user-friendly remote with step-by-step instructions
- Automated priming
- Simplified filling
- Mindful alerts
- Advanced Bluetooth® connectivity



Not actual size or scale.

If you currently are using another pump, speak to your healthcare team to learn more about RemunityPRO.

IMPORTANT SAFETY INFORMATION FOR REMUNITYPRO (CONT'D)

Warnings and Cautions (Cont'd)

Do not use disposables from previously opened or damaged sterile packaging, damaged disposable components, or expired sterile components.

IMPORTANT SAFETY INFORMATION FOR REMODULIN (CONT'D)

Before you take Remodulin, tell your healthcare provider if you: (Cont'd)

- Are taking gemfibrozil (for high cholesterol), rifampin (for infection) or other drugs that affect liver enzymes. Your doctor may need to adjust your Remodulin dosage.

Please see Important Safety Information for RemunityPRO and Remodulin on page 8-9 and the Full Prescribing Information and Patient Information for Remodulin in pocket.

Getting started with RemunityPRO™

Take the first step forward with confidence

Before you start, familiarize yourself with the features that **RemunityPRO offers** and learn how to use this beginner-friendly pump, including quickly learning to fill and change cassettes. The user-friendly remote provides step-by-step guidance and real-time feedback, while educational videos provide support when you need it.



Scan here to explore the **RemunityPRO features**

When you're ready to start, **MyRemodulin.com** provides a personalized website experience from day one. It offers information specific to your therapy's route of administration, your pump, and your journey. You'll have access to pump user guides, encouraging videos, helpful tips for an improved experience, and much more.



Scan here to watch an educational video about **MyRemodulin.com**



IMPORTANT SAFETY INFORMATION FOR REMUNITYPRO (CONT'D)

Warnings and Cautions (Cont'd)

If the pump or remote fail to work as detailed in the user guide, stop using the pump and remote and switch to the alternate pump and remote included with your system.

IMPORTANT SAFETY INFORMATION FOR REMODULIN (CONT'D)

Before you take Remodulin, tell your healthcare provider if you: (Cont'd)

- Are pregnant, breastfeeding, or planning to become pregnant. It is not known if Remodulin will harm your unborn baby or if Remodulin passes into your breast milk.



A treatment experience you can be confident in from your very first day

Tips for staying on track with Remodulin

It may be intimidating at first, but you've got this! Remember you are not alone. Keep in touch with your healthcare team and lean on your community.

Your **Specialty Pharmacy** is an essential partner in your treatment journey. They can help with access to medication and supplies, in-home training, follow-up visits, and a 24-hour hotline.



Scan here to access **Specialty Pharmacy** on our website

Remodulin has **active social communities** on Facebook, Instagram, and YouTube that allow you to connect with other people dealing with the same issues you are facing.



Scan here to join the **Facebook community**



“

Seek out a support group so you can share your fears, daily problems, and joys.”

—PAH patient

IMPORTANT SAFETY INFORMATION FOR REMUNITYPRO (CONT'D)

Warnings and Cautions (Cont'd)

Only use the supplied cables with your RemunityPRO system as other cables could interfere with pump function.

IMPORTANT SAFETY INFORMATION FOR REMODULIN (CONT'D)

What are the serious side effects of Remodulin?

- Continuous intravenous (IV) infusions of Remodulin delivered using an external infusion pump, with a tube placed in a central vein within the chest, are associated with the risk of blood stream infections and sepsis, which may be fatal. Therefore, continuous subcutaneous (SC) infusion delivered just beneath the skin is the preferred type of delivery.

Please see **Important Safety Information for RemunityPRO and Remodulin on page 8-9** and the **Full Prescribing Information and Patient Information for Remodulin in pocket.**

Access and support from day one



What you need, when you need it most

United Therapeutics Cares provides you with ongoing, one-on-one education and support beginning on day one of your United Therapeutics treatment journey.

Your dedicated Patient Navigator will personalize support and answer questions about what to expect as you begin treatment, including insurance coverage, out-of-pocket cost assistance, co-pay assistance, and prescription fulfillment.

If you have any questions, call 1-844-864-8437 to speak with your Patient Navigator, Monday through Friday, 8:30 am - 7 pm ET.



Visit [UnitedTherapeuticsCares.com](https://www.UnitedTherapeuticsCares.com) to learn more and enroll today

IMPORTANT SAFETY INFORMATION FOR REMUNITYPRO (CONT'D)

Warnings and Cautions (Cont'd)

Disconnect the infusion set tubing from your infusion site before removing the cassette from the pump.

IMPORTANT SAFETY INFORMATION FOR REMODULIN (CONT'D)

What are the serious side effects of Remodulin? (Cont'd)

- Worsening of PAH symptoms. Do not stop taking or greatly reduce your Remodulin dose without consulting your doctor.



RemunityPRO™ Pump for Remodulin® (treprostinil) Injection

Indication

The RemunityPRO Pump for Remodulin (treprostinil) Injection is intended for continuous subcutaneous delivery of Remodulin (treprostinil) Injection for use in patients, ages 17 and older.

Important Safety Information for RemunityPRO

Warnings and Cautions

Do not use the system outside the conditions listed in the User Guide. Retain the User Guide for future reference. Refer to the User Guide for all warnings and cautions. Failure to comply with the following warnings and cautions may result in harm.

Limited to use with Remodulin. Only RemunityPRO cassettes may be used with the RemunityPRO pump. RemunityPRO pump is for use only with FDA-cleared infusion sets: Medtronic Quick-set Infusion Set (MMT-392, MMT-393), Neria Guard (704060-5229, 704060-5226), Medtronic Silhouette Infusion Set (MMT-373), and Smiths Medical Cleo 90 Infusion Set (21-7230-24, 21-7220-24).

Do not use disposables from previously opened or damaged sterile packaging, damaged disposable components, or expired sterile components. If the pump or remote fail to work as detailed in the user guide, stop using the pump and remote and switch to the alternate pump and remote included with your system. Only use the supplied cables with your RemunityPRO system as other cables could interfere with pump function. Disconnect the infusion set tubing from your infusion site before removing the cassette from the pump. Avoid exposure of your pump and cassette to temperatures below 41°F (5°C) or above 104°F (40°C). The pump may affect nearby electrical and electronic devices, including medical devices, cell phones, Bluetooth devices, RFID readers, Wi-Fi equipment, and strong magnetic fields causing these devices to operate abnormally or to stop functioning. Keep the pump, remote, and battery charger at least 12 inches (30 cm) away from electrical, medical, and wireless devices. Do not open, crush, heat above 140°F (60°C), or incinerate the pump battery or remote, as doing so can lead to fire, burns, and property damage. This system supports flow rates between 8 µL/h and 225 µL/h. If the required flow rate is outside this range, discuss with your Specialty Pharmacy or healthcare provider.

Prescription Information

Caution: Federal law (USA) restricts this device to sale by or on the order of a licensed healthcare provider. Use of this device without the training and supervision of a healthcare provider may lead to errors that result in harm.

REMUPROISIconMAR2025

See the RemunityPRO Pump for Remodulin (treprostinil) Injection User Guide for further detailed important safety information including warnings, cautions, and instructions on how to properly use the system.

For further information, please call United Therapeutics Corp. at 1-844-864-8437.

The RemunityPRO Pump for Remodulin (treprostinil) Injection is manufactured for United Therapeutics Corp.

You may report side effects to the FDA at www.fda.gov/MedWatch or call 1-800-FDA-1088.

Remodulin® (treprostinil) Injection

What is Remodulin?

Remodulin is a prescription medication used to treat adults with pulmonary arterial hypertension (PAH; WHO Group 1), which is high blood pressure in the arteries of your lungs. Remodulin can reduce symptoms associated with exercise. Remodulin was studied mainly in patients with NYHA Functional Class II-IV symptoms. It is not known if Remodulin is safe and effective in children.

In people with PAH who need to switch from epoprostenol, Remodulin is approved to slow the worsening of symptoms.

Important Safety Information for Remodulin

Before you take Remodulin, tell your healthcare provider if you:

- Have other medical conditions or take other medicines that may affect your use of Remodulin by increasing the risk of side effects or decreasing the drug's effectiveness.
- Have liver problems. Your Remodulin dose may need to be adjusted if you have liver problems.
- Have low blood pressure or bleeding problems.
- Are taking gemfibrozil (for high cholesterol), rifampin (for infection) or other drugs that affect liver enzymes. Your doctor may need to adjust your Remodulin dosage.
- Are pregnant, breastfeeding, or planning to become pregnant. It is not known if Remodulin will harm your unborn baby or if Remodulin passes into your breast milk.

What are the serious side effects of Remodulin?

- Continuous intravenous (IV) infusions of Remodulin delivered using an external infusion pump, with a tube placed in a central vein within the chest, are associated with the risk of blood stream infections and sepsis, which may be fatal. Therefore, continuous subcutaneous (SC) infusion delivered just beneath the skin is the preferred type of delivery.
- Worsening of PAH symptoms. Do not stop taking or greatly reduce your Remodulin dose without consulting your doctor.
- Low blood pressure (symptomatic hypotension). If you have low blood pressure or are taking drugs that lower your blood pressure, the risk of low blood pressure is increased.
- Bleeding problems. Remodulin may increase the risk of bleeding in people who take blood thinners (anticoagulants).

What are the possible side effects of Remodulin?

- In clinical studies of SC infusion of Remodulin, most people experienced infusion site pain and infusion site reaction (redness, swelling, and rash). These symptoms were sometimes severe and sometimes required treatment with narcotics or discontinuation of Remodulin.
- IV infusion of Remodulin delivered through an external pump has been associated with the risk of blood stream infections, arm swelling, tingling sensations, bruising, and pain.
- The most common side effects seen with either SC or IV Remodulin were headache, diarrhea, nausea, rash, jaw pain, widening of the blood vessels (vasodilatation), and swelling from fluid retention (edema). These are not all the possible side effects of Remodulin. Call your doctor for medical advice about side effects.

You may report side effects to the FDA at www.fda.gov/MedWatch or call 1-800-FDA-1088.

The risk information provided here is not comprehensive. To learn more about Remodulin, talk with your healthcare provider. Please see Full Prescribing Information at www.remodulin.com or call Customer Service at 1-844-UNITHER (1-844-864-8437).

REMISIconFEB2025

RemunityPRO™:

Innovation that delivers



Convenience

Compact and lightweight at 2" in diameter, 3/4" thick, and 1.7 ounces, RemunityPRO fits easily on a belt or in a pocket. Easy to learn, easy to fill, and auto-priming, it delivers 72 hours of medication for peace of mind.



Confidence

Adjusting to pump therapy can take time, but RemunityPRO—designed with feedback from patients with PAH—makes it more simple. It features optimized Bluetooth®, a redesigned remote with guided instructions, streamlined alerts, and more, to help you feel confident and supported.



Care and Support

You'll have a team of experts and a range of resources dedicated to supporting you and making sure you have what you need, when you need it. Throughout your journey, RemunityPRO helps you focus on what matters most.



Scan here to visit our website and learn more about RemunityPRO



Scan here to sign up for MyRemodulin.com patient portal

IMPORTANT SAFETY INFORMATION FOR REMUNITYPRO (CONT'D)

Warnings and Cautions (Cont'd)

Avoid exposure of your pump and cassette to temperatures below 41°F (5°C) or above 104°F (40°C).

IMPORTANT SAFETY INFORMATION FOR REMODULIN (CONT'D)

What are the serious side effects of Remodulin? (Cont'd)

- Low blood pressure (symptomatic hypotension). If you have low blood pressure or are taking drugs that lower your blood pressure, the risk of low blood pressure is increased.

Please see Important Safety Information for RemunityPRO and Remodulin on page 8-9 and the Full Prescribing Information and Patient Information for Remodulin in pocket.

Remodulin is a registered trademark and RemunityPRO is a trademark of United Therapeutics Corporation. All other brands are trademarks of their respective owners. The makers of these brands are not affiliated with and do not endorse United Therapeutics or its products.

© 2025 United Therapeutics Corporation. All Rights Reserved.
US/REM/1034 Printed in USA.

