



# Elexy™

Redefining possibilities



Volume

Viscosity



Device Type	Reusable electromechanical autoinjector
Primary Container	1.0 mL and 2.25 mL pre-filled syringe
Usage	Single dose   Reusable power unit
Dosage Type	Empty-all   Full contents
Route of Administration	Subcutaneous
Activation Method	Needle cover
Mode of connectivity	Cellular (LTE) with optional Bluetooth® (BLE)
Injection Feedback	Audio, continuous visual (drug inspection window and LED light), and tactile feedback

Copyright © 2025 SHL Medical AG. All rights reserved. Version 1.1 Information accurate at time of print and is subject to change without notice. Contact [info@shl-medical.com](mailto:info@shl-medical.com) for more information.

## Handling Instructions



Insert PFS drug cassette into power unit, twist until dots align.



After top light turns on, uncap.



Press down on needle cover and wait until top light fades, indicating injection completion.



Remove drug cassette and dispose into sharp container.


## Key Features

- ✓ One interface that is compatible to both pre-filled syringes and cartridges.
- ✓ Features a reusable and rechargeable power unit.
- ✓ Adaptable device for flexible drug volumes, drug viscosities, and injection speeds.
- ✓ Integrated cellular connectivity for seamless injection experience (Bluetooth® optional).
- ✓ Multiple injection feedback, including visual, audio, and tactile.



# Elexy™

Redefining possibilities

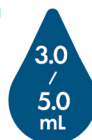

Volume   
Viscosity 

Device Type	Reusable electromechanical autoinjector
Primary Container	3.0 mL and 5.0 mL cartridge with Needle Isolation Technology (NIT®)
Usage	Single dose   Reusable power unit
Dosage Type	Empty-all   Full contents
Route of Administration	Subcutaneous
Activation Method	Needle cover
Mode of connectivity	Cellular (LTE) with optional Bluetooth® (BLE)
Injection Feedback	Audio, continuous visual (drug inspection window and LED light), and tactile feedback

Copyright © 2025 SHL Medical AG. All rights reserved. Version 1.1 Information accurate at time of print and is subject to change without notice. Contact [info@shl-medical.com](mailto:info@shl-medical.com) for more information.

## Handling Instructions



1 Insert cartridge drug cassette into power unit, twist until dots align.



2 After top light turns on, unscrew cap.



3 Press down on needle cover and wait until top light fades, indicating injection completion.



+ Remove drug cassette and dispose into sharp container.

## Key Features

- ✓ One interface that is compatible to both pre-filled syringes and cartridges.
- ✓ Features a reusable and rechargeable power unit.
- ✓ Adaptable device for flexible drug volumes, drug viscosities, and injection speeds.
- ✓ Integrated cellular connectivity for seamless injection experience (Bluetooth® optional).
- ✓ Multiple injection feedback, including visual, audio, and tactile.