



Maggie®

Two Step Cartridge-Based Solution



Volume



Viscosity



5.0
mL

Device Type	Disposable autoinjector
Primary Container	5.0 mL cartridge with pre-installed needle (NIT®)
Dosage Type	Fixed dose Optional automated priming and titration
Route of Administration	Subcutaneous Intramuscular
Activation Method	Needle cover
Injection Feedback	Continuous audible clicks during injection, viewing window

Copyright © 2025 SHL Medical AG. All rights reserved. Version 1.1 Information accurate at time of print and is subject to change without notice. Contact info@shl-medical.com for more information.

Handling Instructions

* For suspension drugs that require a priming step – hold device with cap upright during cap removal.



1
Unscrew cap until needle cover automatically extends*. Then remove cap.



2
Press and hold against injection site to activate.



3
Discard in sharps container.

Key Features

- ✓ Simple 2-step operation
- ✓ Needle cover activation
- ✓ Continuous audible and visual feedback
- ✓ One-handed operation
- ✓ Permanently hidden needle
- ✓ Compact design
- ✓ Customizable needle length and gauge
- ✓ Passive locking needle cover
- ✓ Customized to client specifications