



Volume 
Viscosity 

Device Type	Disposable autoinjector
Primary Container	Dual-chamber cartridge Fill volume between 0.5–0.7ml*
Dosage Type	Fixed Dose
Route of Administration	Subcutaneous
Activation Method	Needle cover
Injection Feedback	Audible click at beginning of injection, viewing window

Copyright © 2025 SHL Medical AG. All rights reserved. Version 1.0 Information accurate at time of print and is subject to change without notice. Contact info@shl-medical.com for more information.
*Fill volume below 0.5 ml requires additional feasibility testing.

Handling Instructions



Unlock the knob to initiate automated reconstitution.



Twist off the needle cap to automatically open fluid path and initiate priming.



Push the device against the injection site to begin the injection. Wait for the plunger rod to stop moving which indicates end of injection.



Discard in sharps container.

Key Features

- ✓ 3-step operation
- ✓ Automatic reconstitution
- ✓ Supports delivery of lyophilized formulations
- ✓ Cap is locked prior to reconstitution
- ✓ Pre-attached Needle Isolation Technology (NIT®)
- ✓ Customizable cannula gauge and length
- ✓ Eliminates manual needle attachment
- ✓ Needle cover activation
- ✓ Permanently hidden needle