

Considerations for Equipment and Supplies for Sedation-Free TNE with the EvoEndo[®] System

Many of the items below may already be available in your department. Some may need to be purchased and some might be shared with other staff and departments in the hospital. For complete product information, warnings, and instructions, please read our Instructions for Use at evoendo.com/ifu.

Ideal Setup for Sedation-Free Endoscopy Room

Moderately sized, child-friendly room with adequate ventilation

- Seating for patient: reclining/height adjustable chair and armrests
(i.e. ENT-style chair)
- Seating for family
- General storage cabinets
- Endoscope storage cabinets
- Telephone
- Dimmable lights
- Sink and hand wash station
- Biohazard/sharp disposal

Endoscopy Equipment

- EvoEndo[®] Controller with HDMI Cable and Power Supply *(both included)*
- Pressurized air or carbon dioxide source
Refer to the Instructions for Use and work with the EvoEndo[®] Clinical Team for optimal settings for your site.
- Medical suction source (200 mmHg)
(available from EvoEndo)
- Medical grade monitor with 1080p compatibility *(available from EvoEndo)*
Refer to the Instructions for Use and work with the EvoEndo[®] Clinical Team for optimal monitor recommendations for your site.
- Medical monitor stand/mount/cart
(available from EvoEndo)

Phone compatible with [Google Cardboard](#), WiFi, or Cellular Capabilities, and HD Screen

EvoEndo[®] Recommends: Samsung S9 or later generation phone; Google Pixel 2 or later phone. Patients can also use their own phones if compatible. Screen Pixel Density > 450 ppi.

- Wi-Fi with high-speed internet access
- Access to EvoEndo[®] YouTube Channel:
evoendo.com/video

Computer system with endoscopic reporting software and color printer

- Medical/endoscopy cart or stand
(available from EvoEndo)
- Power strip
- Backup power supply
- Image capture reporting software integration adapters *(optional)*

TNE RN/tech work area

- Illumination and a biopsy specimen tray
- Bag valve mask
- Blood pressure monitor equipment
- Pulse oximeter

Special Equipment

- Code cart

Continued on next page

Single-Use Procedure Supplies for Sedation-Free TNE with EvoEndo

Endoscopy Supplies

- EvoEndo[®] Model LE Single-Use Gastroscope
- Biopsy forceps or other accessory for 2 mm x 1.1 m endoscopy channel
- Biopsy specimen containers
(4-6 depending on procedure)
- Specimen labels
- Water-based medical lubricant jelly
- 250-500 ml sterile water bottle with 1.25" standard endoscopy cap
- Anti-fog solution
- 2"x2" gauze
- Suction connecting tubing

Patient Distraction and Dissociation Supplies

- EvoEndo[®] Patient Experience Kit with single-use VR goggles and squeeze ball

Patient Care Supplies

- Emesis bag
- Water rinse cup (*sterile preferred*)
- Wash towels
- Nasal tissue
- Burp cloths
- Slush, juice, or popsicle

Medication

- 4% lidocaine solution
- Viscous lidocaine
- Oxymetazoline
- Nasal spray atomizer
- Simethicone liquid or gelcap

Personal Protective Equipment

- Gloves (*provider preference*)
- Gowns
- Face shield
- Face masks or N95 per protocols at hospital

Room Cleaning Supplies

- Medical grade cleaning solution/wipes per hospital protocol

Recycling

- Special recycling programs for single-use endoscopes may be available through a hospital's existing recycling programs for similar devices or medical waste. Contact EvoEndo for recommendations.

 *Included in the EvoEndo[®] System*