

 **UTM**[®] UNIVERSAL TRANSPORT MEDIA

COLLECTION, TRANSPORT, AND PRESERVATION OF
CLINICAL SPECIMENS FOR VIRAL MOLECULAR DIAGNOSTIC
TESTING

 **HARDY**
DIAGNOSTICS
A Culture of Service[™]

ROOM TEMPERATURE
STABLE



CLEARED FOR
USE WITH VIRAL
MOLECULAR ASSAYS



MAINTAINS VIABILITY
OF ORGANISMS FOR
UP TO 48 HOURS




CONTAINS
ANTIBIOTICS TO
INHIBIT BACTERIAL
AND FUNGAL FLORA
IN PATIENT SAMPLES

UTM™ Collection Quick Guide




COLLECT

- Collect the patient sample using the swab. To limit the risk of contamination, be sure that the swab tip only touches the collection site and avoid touching the swab below the breakpoint.

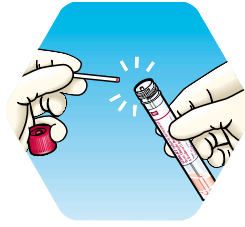

SNAP

- Remove cap from the tube and insert the swab to the bottom of the tube.
- Holding the tube away from face, grasp the end of swab shaft and bend at a 180-degree angle to break at the pink breakpoint. If needed, gently rotate the swab shaft to complete the breakage.
- Screw the cap on tightly to prevent leakage.

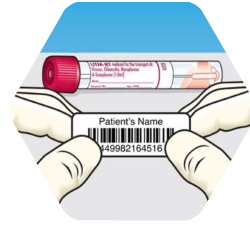



SEND

- Identify tube with patient information and send to laboratory.

Specimen collection should be performed by health care personnel who have completed training and demonstrated competency. Always read the manufacturer's package insert for specific instruction regarding specimen collection and transport for the type of test kit being used.

Product Description

CAT. NO.	DESCRIPTION	UNIT
3C047N	 3 mL UTM® Medium in 16x100mm Skirted Tube with Plastic Red Capture Cap - Individually Packaged, Sterile	50/Pack; 6 Packs/Case
1U054S01	 Single Regular Size Polyester Swab with Breakpoint - Individually Packaged, Sterile	100/Pack; 10 Packs/Case

Please note that products 3C047N (UTM) and 1U054S01 (Swabs) must be ordered together.

Add them both to your purchase order.

- UTM's Minimum Order: 10 Packs (300 UTM's per Pack)
- Order in Increments of 10 only

Swabs (3 case/1000 swabs per case) are Included FREE with your order of 3C047N UTM's. To take advantage of the free swabs please add 3 x 1U054S01 to your Purchase Order for every quantity of 10x 3C047N purchased. Price will be \$0.00 each.

If you prefer NP flocked swabs, you can buy these separately, Cat. no. 972012 (PK100), not free.



Headquarters
1430 West McCoy Lane
Santa Maria, CA 93455
800.266.2222
Sales@HardyDiagnostics.com
HardyDiagnostics.com



Distribution Centers
Santa Maria, California
Olympia, Washington
Salt Lake City, Utah
Phoenix, Arizona
Dallas, Texas
Springboro, Ohio
Lake City, Florida
Albany, New York
Raleigh, North Carolina

Hardy Diagnostics adheres to cGMP, is licensed by the FDA as a medical device manufacturer, and its quality management system is ISO 13485 certified.

Copyright © 2020 Hardy Diagnostics

091020sg