



Status H. pylori

Rapid immunoassay for the qualitative detection of anti-Helicobacter pylori antibodies in human whole blood, serum or plasma



- **CLIA Waived** for whole blood
- Sensitivity of 95.9% with 91.6% overall agreement to biopsy histology
- Easy-to-use single reagent procedure
- Results available in 10 minutes
- Requires only 25µl of whole blood or 10µl serum/plasma
- Convenient finger stick sample collection

CPT CODE


86318QW
86318

Whole Blood
Serum/Plasma

Status H.pylori


Refer to Product Instructions for details.

STEP 1
Remove a test device from its pouch and place on a flat surface.

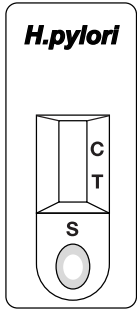


STEP 2
Collect sample using the capillary tube provided and add to test device Sample Well.


- 25µL (red line) for whole blood
- 10µL (black line) for serum/plasma

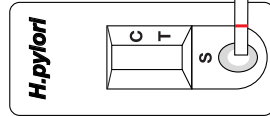



STEP 3
Add 2-3 drops of Developer Solution into Sample Well.

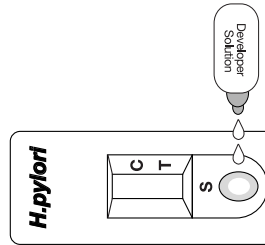

STEP 4
Read the results at 10 minutes.

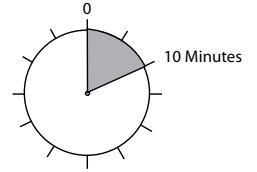


INTERPRETATION OF RESULTS

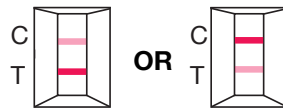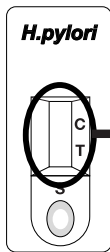


Two Lines Positive (+)

Any visible T line is positive

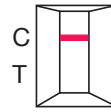

C Line only Negative (-)

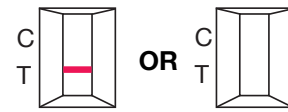

No C Line Invalid

Repeat with new sample and device

ORDERING INFORMATION

PRODUCT	# OF TESTS	ITEM #
Status H. Pylori(whole blood,serum or plasma)	30	37030
CONTROL PRODUCT	# OF TESTS	ITEM #
Status H. Pylori Control Set	2 x 2 ml	200013



Strep A • Women's Health • Drugs of Abuse • Cardiac Markers
iFOBT • Flu A&B • Fertility • Bladder Cancer Marker



Proudly made in the USA.

For more information or to place an order, contact



1430 West McCoy Lane • Santa Maria • California • 93455
800.266.2222 • Sales@HardyDiagnostics.com • HardyDiagnostics.com

Official Distributor of

lifeSign

Skillman, NJ



Scan to find your local Hardy Sales Rep