


Hardy**CHROM**TM

Group A Strep Agar

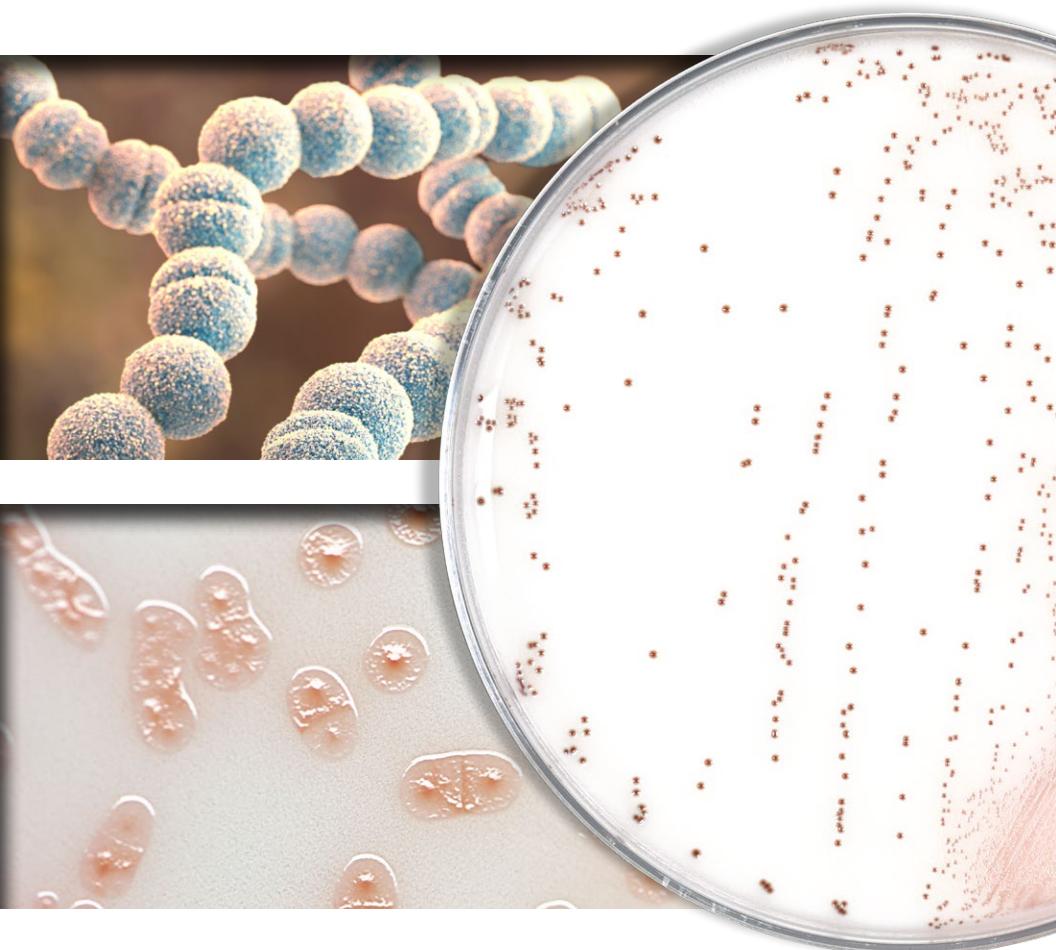


Hardy**CHROM**TM Group A Strep Agar

Color read-out on a white background in just 24 hours.

We are making identification easy with the release of the Hardy Diagnostics HardyCHROM™ Group A Strep Agar. This easy-to-read chromogenic medium is recommended for the selective cultivation and differential isolation of Group A *Streptococcus* (*S. pyogenes*). Colonies are identified based on colony color (red, red-brown, or red-orange colonies) among the other non-GAS bacteria in the complex throat flora (blue, clear or white colonies) after 24 hours of incubation.

15x100mm plate, 10/pk.....**G337**

Reduced stacking ring,
15x100mm plate, 10/pk.....**GA337**

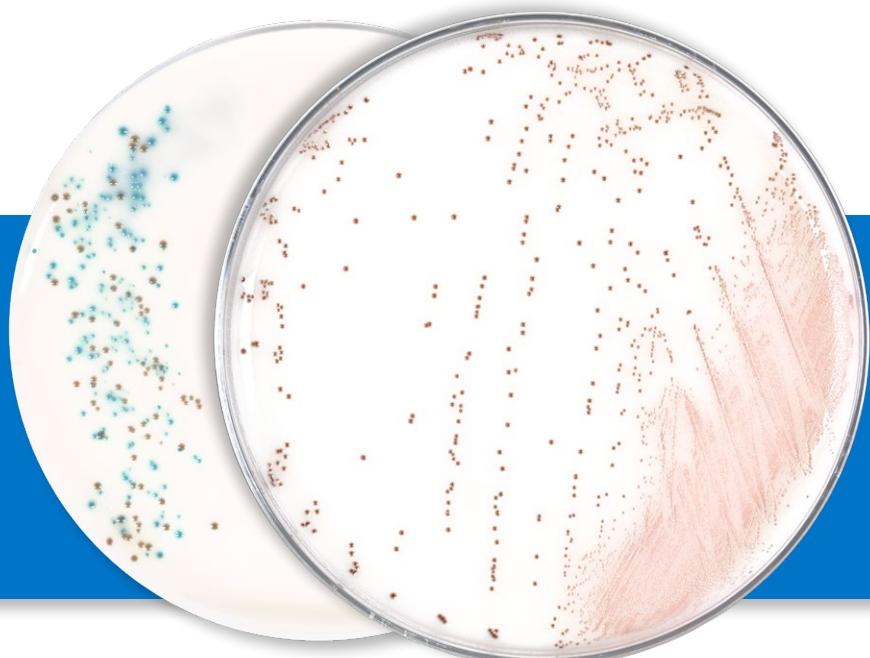


GROUP A STREP infections can occur any time during the year. However, infections are more common in the United States in certain seasons:


- Strep throat and scarlet fever are more common in the winter and spring.
- Impetigo is more common in the summer.

Anyone can get a **GROUP A STREP** infection, but some infections are more common in certain age groups:

- Strep throat and scarlet fever are most common in children between the ages of 5 and 15 years.
- Impetigo is most common in children between the ages of 2 and 5 years. (source CDC)

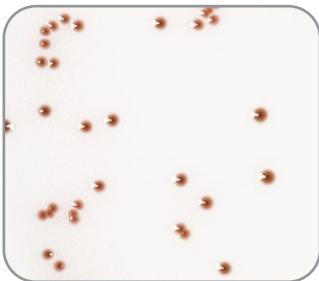

Streptococcus pyogenes (ATCC® 19615)
red-brown colonies growing on
HardyCHROM™ Group A Strep Agar.
Incubated in CO₂ for 24 hours at 35°C.

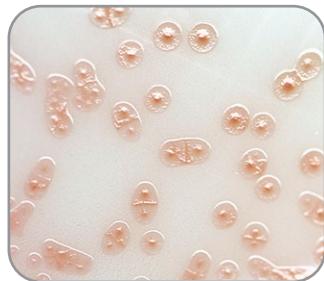

Streptococcus pyogenes (clinical strain)
red-orange colonies growing on
HardyCHROM™ Group A Strep Agar.
Incubated in CO₂ for 24 hours at 35°C.

Did you know that Hardy Diagnostics offers you free blok-boxes for all your light sensitive media? Just ask us if you would like a free blok-box added to your order. One box holds a sleeve of ten Petri plates.





*A Culture of Service*TM



Hardy Diagnostics
donates 1% of each
sale to charity.



FM 572526

Hardy Diagnostics has a Quality Management System that is certified to ISO 13485 and is a FDA licensed medical device manufacturer.

Headquarters

1430 West McCoy Lane
Santa Maria, CA 93455
800.266.2222
Sales@HardyDiagnostics.com
HardyDiagnostics.com

Distribution Centers

Santa Maria, California
Olympia, Washington
Salt Lake City, Utah
Phoenix, Arizona
Dallas, Texas
Springboro, Ohio
Lake City, Florida
Albany, New York
Raleigh, North Carolina

