

MYCOFAST[®] US

UROGENITAL MYCOPLASMA & UREAPLASMA DIAGNOSIS



Detection, identification
and enumeration in
one simple test

Make your *Urogenital diagnosis* eas

MYCOFAST® US has been designed for the detection, identification, and enumeration of *Ureaplasma urealyticum* (U.u.) and *Mycoplasma hominis* (M.h.) in endocervical, urethral, urinary, and semen specimens.



Our exclusive growth activator, the M.h. Supplement, guarantees fast and accurate results.


MYCOFAST® US enables the enumeration of U.u. > 10³ CCU/mL which is compliant with pathological thresholds recognized for semen and urine samples.

1 Inoculation of the lyophilized transport medium



Simpler, faster, and more cost effective!

- All-in-one test
- Ready-to-use reagents, “all-liquid” method
- High sensitivity and specificity
- Fast results for clinicians, available within 24 hours
- Visual colorimetric method, easy interpretation

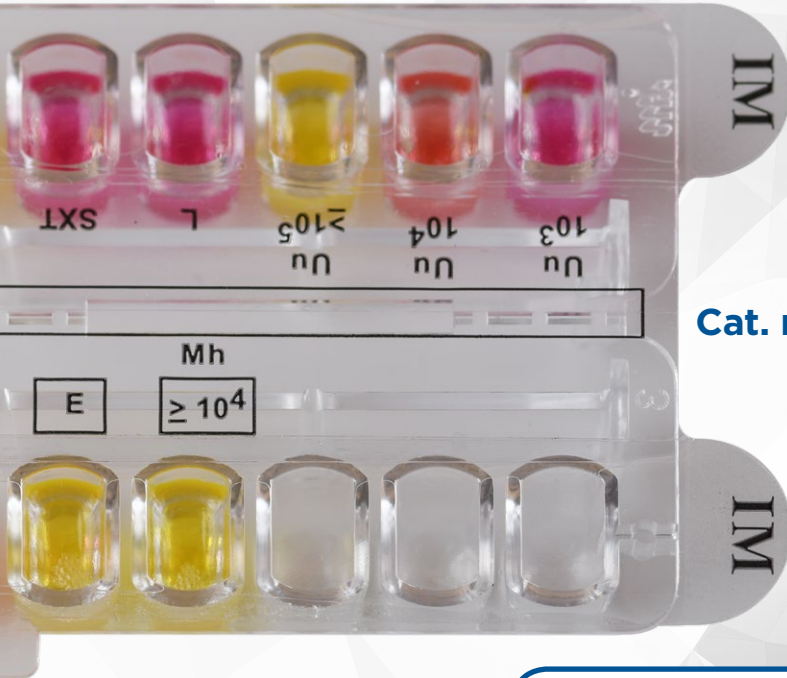

Cat. no. 00050

2

Inoculation of the MYCOFAST® US tray

Enumeration of *M. hominis*

Enumeration of *U. urealyticum*

Identification of *U. urealyticum* and *M. hominis*

3 Interpretation

1 2 3 4 5 6 7



U. urealyticum at 10^4 CCU/ml



M. hominis at $\geq 10^4$ CCU/ml



U. urealyticum at 10^5 CCU/ml *M. hominis* at $\geq 10^4$ CCU/ml

MYCOFAST[®] US

UROGENITAL MYCOPLASMA & UREAPLASMA DIAGNOSIS

Manufactured by ELITech Microbio



HARDY
DIAGNOSTICS
A Culture of Service™



100%
Employee Owned

ESTABLISHED

1980



Made in the USA

Headquarters

1430 West McCoy Lane
Santa Maria, CA 93455
800.266.2222 : phone
Sales@HardyDiagnostics.com
HardyDiagnostics.com/QuickSlide

Distribution Centers

Santa Maria, California
Olympia, Washington
Salt Lake City, Utah
Phoenix, Arizona
Dallas, Texas
Springboro, Ohio
Lake City, Florida
Albany, New York
Raleigh, North Carolina

Copyright © 2019 Hardy Diagnostics

050219ss