



# SYSTEM OF NARCOTICS IDENTIFICATION

## INSTRUCTIONS FOR USE

### TEST R – Valium®

**FOR:** Valium®, Rohypnol® & Methcathinone

#### Test Function

1. This reagent system presumptively identifies Valium, Rohypnol and Methcathinone.

#### How to Use Test R

1. Remove clip and insert into the test pack an amount of powdered suspect material that would fit inside this circle.  Reseal with clip and tap gently to assure material falls to the bottom of the pouch.
2. With the printed side of the test pouch facing you, break the ampules from left to right by squeezing the center of the ampule with the tips of the thumb and forefinger.
3. Break the left ampule containing the white particles. Be sure the particles fall to the bottom of the test pouch.
4. Break the right ampule and agitate. A light violet/purple color will begin to develop almost immediately. After about (1) minute, a violet/purple color closely matching the test pouch color swatch will develop, should Valium, Rohypnol or Methcathinone be present.

#### Interpretation of Resulting Colors

1. If the reagent in solid form is slow in dissolving, the violet color will be slow in developing but should appear in approximately (1) minute after the right ampule has been broken, providing agitation is applied continuously.

Valium® and Rohypnol® are registered trademarks of Roche Products, Inc.

### CAUTION

- A. Since these tests contain strong acids, it is suggested that Test F - Acid Neutralizer be used after testing and before disposal.
- B. Before discarding used test packs, remove clip and add one measure of acid neutralizer from Test F.
- C. Add slowly to prevent bubbling over.
- D. DO NOT reseal the test pouch until effervescence has completely stopped.
- E. Reseal pouch with clip and discard in a tamper-free disposal unit.
- F. No attempt should be made to crush the glass particles after the ampoules are broken.
- G. **Antidote:** In case of contact, immediately flush eyes or skin with water for at least 15 minutes while removing contaminated clothing and shoes. If swallowed, do not give emetics. Contact a physician.

### WARNING

These tests are NOT designed for use with liquid samples.



13386 International Parkway  
Jacksonville, Florida 32218  
904.485.1836 FAX: 904.741.5407  
© 2105 Safariland, LLC

A Brand of the Safariland Group