



SENNHEISER

Sonite R

User guide

Sennheiser Sonite R

Sennheiser Sonite R Loaner

Sennheiser S Charger R



This user guide is valid for:

Wireless hearing aids

Sennheiser Sonite R

Sennheiser Sonite R Loaner



Charging accessory

Sennheiser S Charger R



Note that instructions for the activity tracking functions can be found in the Easy Line Remote™ app user guide.



Your hearing aid and charger details

- i** If no box is checked and you do not know the model of your hearing aid or charging accessories, please ask your hearing care professional.
- i** The hearing aids described in this user guide contain a built-in and non-removable rechargeable lithium-ion battery.
- i** Please also read safety information regarding handling rechargeable hearing aids (chapter 24).

Hearing aid models

- Sennheiser Sonite R
- Sennheiser Sonite R Loaner

Earpieces

- Dome
- SlimTip
- cShell

Charging accessory

- Sennheiser S Charger R

These premium products are the result of decades of research and expertise, and are designed to keep you connected to the beauty of sound! We thank you for making such a great choice and wish you many years of listening pleasure.

Please read the user guide carefully to make sure that you understand and get the most out of your hearing aids. For more information about features and benefits, simply contact your hearing care professional.

Training is not required for handling of this device. A hearing care professional will help set up this device according to your individual preferences during the fitting consultation.

For more information regarding features, benefits, set up, use, maintenance or repairs of your hearing aid and accessories, please contact your hearing care professional or the manufacturer representative. Additional information can be found in the datasheet of your product.

Contents

Your hearing aid and charger

- 1. Quick guide 7
- 2. Parts of the hearing aid and charger 9

Using the charger

- 3. Preparing the charger 13
- 4. Charging the hearing aids 15

Using the hearing aid

- 5. Left and right hearing aid markings 19
- 6. Putting on the hearing aid 20
- 7. Removing the hearing aid 21
- 8. Multi-function button 22
- 9. On/Off 23
- 10. Touch Control 24
- 11. Connectivity overview 25
- 12. Initial pairing 26
- 13. Phone calls 28
- 14. Flight mode 33
- 15. Rebooting your hearing aid 36
- 16. Easy Line Remote™ app overview 37

Further information

| | |
|--|----|
| 17. Environmental conditions | 38 |
| 18. Care and maintenance | 41 |
| 19. Exchanging the earwax protection | 44 |
| 20. Service and warranty | 49 |
| 21. Compliance information | 51 |
| 22. Information and description of symbols | 60 |
| 23. Troubleshooting | 65 |
| 24. Important safety information | 70 |

1. Quick guide


Charging your hearing aids

i Before using your hearing aid for the first time, it is recommended to charge it for 3 hours.


After placing your hearing aid in the charger, the indicator light on the hearing aid will show the charging state of the battery until it is fully charged. When fully charged the indicator light will be constantly on, i.e. solid green light.



Left & right hearing aid markings

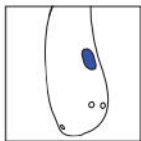

Blue marking for left hearing aid.

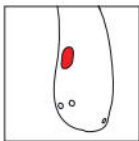

Red marking for right hearing aid.

Multi-function button with indicator light

The button has several functions, the primary function is as an on/off switch. Together with your hearing care professional, you can decide if it also allows for volume control and/or program change. This is indicated in your individual instructions.



Phone calls: If paired with a Bluetooth® enabled phone, a short press will accept and a long press will reject an incoming call.

On/Off: Firmly press the lower part of the button for 3 seconds until the indicator light blinks.

On: indicator light is solid green

Off: indicator light is solid red

Enter flight mode: When device is off, press the lower part of the button for 7 seconds until the solid orange light appears. Then release the button.

Touch Control

If paired with a Bluetooth® enabled device, multiple functions can be accessed using Touch Control, please refer to chapter 10. This is also indicated in your individual instructions. To use Touch Control, tap the top of your ear twice.

2. Parts of the hearing aid and charger

The following pictures show the hearing aid model and charger accessories described in this user guide. You can identify your personal models by:

- checking “Your hearing aid and charger details” on page 3.
- comparing your earpieces, hearing aid and charger with the following models.

Possible earpieces



Dome




SlimTip



cShell


Sennheiser Sonite R

Sennheiser Sonite R Loaner

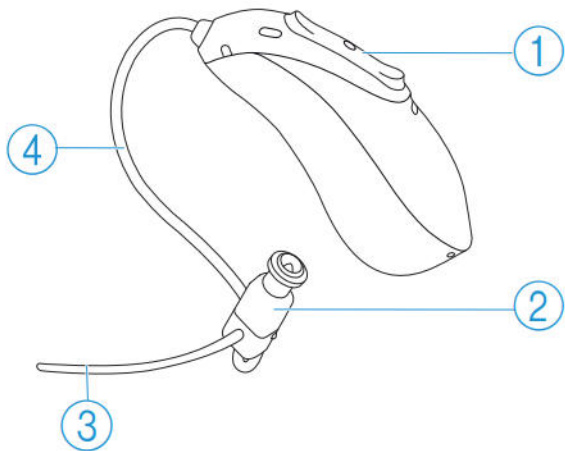


- 1 Multi-function button with built-in indicator light
- 2 Speaker (without earpiece attached)
- 3 Anchor (optional)
- 4 Tube

Sennheiser S Charger R

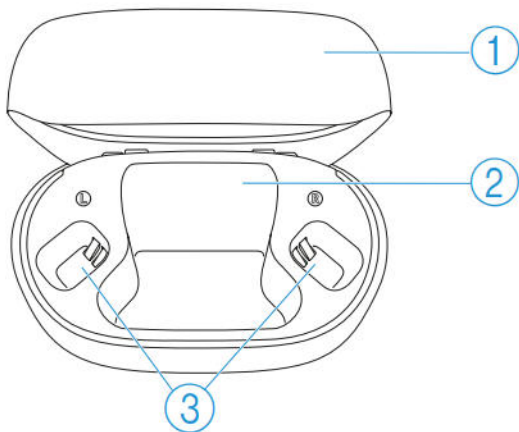


- 1 Charger case lid
- 2 Cavity for earpieces
- 3 Hearing aid charging slots with left/right indicators

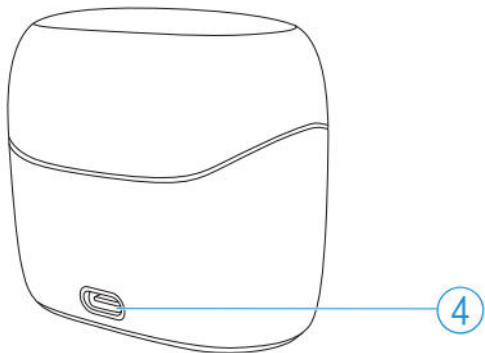

4 USB-C connection to external power supply

External power supply

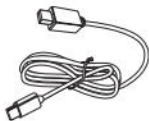


USB cable (< 3 m)



Power supply

3. Preparing the charger

Connecting the power supply

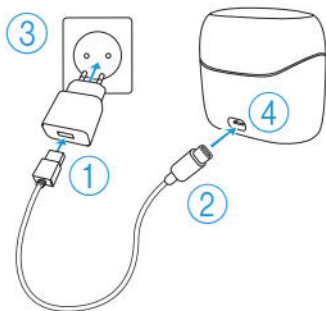

- 1 Plug the larger end of the charging cable into the power supply.
- 2 Plug the smaller end into the USB port on the charger.
- 3 Plug the power supply into a power outlet.
- 4 The indicator light around the USB port will light up green when the charger is connected to the power outlet.

The charger may be used with an external power supply that fulfills the specifications in the table below.

Power supply and charger specifications

| | |
|-----------------------------|--|
| Power supply input voltage | 100-240 VAC, 50/60 Hz, current 0.25 A |
| Power supply output voltage | 5 VDC +/-5%, current 1 A |
| Charger input voltage | 5 VDC +/-5%, current 250 mA |
| USB cable specifications | 5 V min. 1 A, USB-A to USB-C, maximum length 3 m |

4. Charging the hearing aids


- i** Low power: You will hear two beeps when the hearing aid battery is low. You will have approximately 60 minutes before you have to charge the hearing aids, after this time the hearing aid automatically shuts down (this can vary, depending on the hearing aid settings).
- i** The hearing aid contains a built-in and non-removable rechargeable lithium-ion battery.
- i** Before using your hearing aid for the first time, it is recommended to charge it for 3 hours.
- i** The hearing aid must be dry before charging, see chapter 17.
- i** Please ensure to charge and use the hearing aid within operating temperature range: +5 to +40 °Celsius (+41 to +104 °Fahrenheit).

4.1 Using the charger

Please refer to chapter 3 for how to set up the charger.

1.

Insert the hearing aid into the charging slots and the earpieces in the large cavity. Make sure the left and right hearing aid markings match the left (blue), right (red) indicators next to the charging slots. The hearing aids will automatically switch off once inserted into the charger.

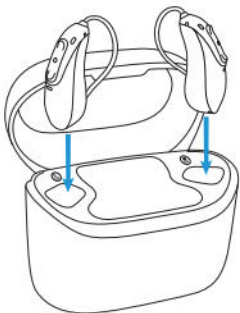





2.

The indicator light on the hearing aid will show the charging state of the battery until the hearing aid is fully charged. When fully charged the indicator light will remain on, i.e. solid green light.

The charging process will automatically stop when the batteries are fully charged, so the hearing aids can be safely left in the charger. It can take up to 3 hours to charge the hearing aids. The charger lid can be closed while charging.

Charging times

| Indicator light | Charging state | Approximate charging times |
|--|----------------|--|
|  | 0 – 10% | |
|  | 11 – 80% | 30 min (30%) 60 min (50%) 90 min (80%) |
|  | 81 – 99% | |
|  | 100% | 3 h |

3.

Remove the hearing aids from the charging slots by simply lifting them out of the charger.

The hearing aid can be set to switch on automatically when removed from the charger. The indicator light starts blinking. The solid green light indicates the hearing aid is ready for use.

If you unplug the charger while the hearing aid is charging inside, please make sure to turn off the hearing aid to prevent discharge. To switch off the charger, remove the power supply from the power outlet.


- i** Turn off your hearing aid before inserting it in the disconnected charger to store it.
- i** Do not hold the tubes to remove the hearing aids from the charger as this can damage the tubes.

5. Left and right hearing aid markings

There is a red or blue marking on the back of the hearing aid and on the speaker. This will tell you if it is a left or a right hearing aid.

Blue marking
for left hearing aid.


Red marking
for right hearing aid.

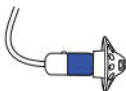

6. Putting on the hearing aid


1.

Place the hearing aid behind your ear.

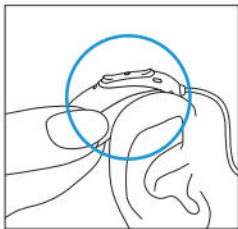


2.

Insert the earpiece into your ear canal.

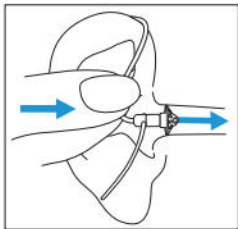


3.

If there is an anchor attached to the earpiece, tuck it into the bowl of your ear to secure your hearing aid.

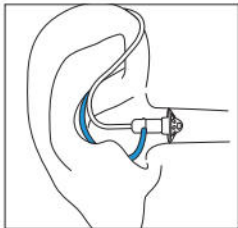


7. Removing the hearing aid

Pull on the bend of the tube and remove the hearing aid from behind the ear.

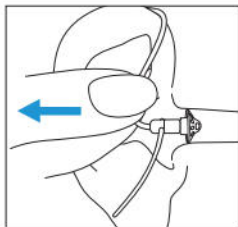

8. Multi-function button

The button has several functions, the primary function is as an on/off switch. Together with your hearing care professional, you can decide if it also allows for volume control and/or program change. This is indicated in your individual “Hearing aid instructions”. Please ask your hearing care professional for a printout.

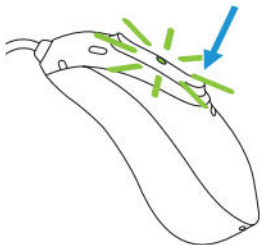


If the hearing aid is paired with a Bluetooth® enabled phone, a short press on the upper or the lower part of the button will accept an incoming call and a long press will reject an incoming call – refer to chapter 13.

9. On/Off

Switch on hearing aid

The hearing aid is configured to switch on automatically when removed from the charger. If this function is not configured, firmly press and hold the lower part of the button for 3 seconds until the indicator light blinks. Wait until the solid green light indicates the hearing aid is ready.

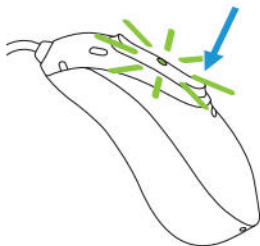

Switch off hearing aid

Press and hold the lower part of button for 3 seconds until the solid red light indicates the hearing aid is switching off.



Blinking green

Hearing aid is switching on



Solid red for 2 seconds


Hearing aid is switching off

i When you switch on the hearing aid you might hear a start-up melody.

10. Touch Control


If paired with a Bluetooth® enabled device, multiple functions can be accessed using Touch Control, e.g. accept/end phone call, pause/resume streaming, start/stop voice assistant. This is indicated in your individual “Hearing aid instructions”.

To use Touch Control, tap the top of your ear twice.

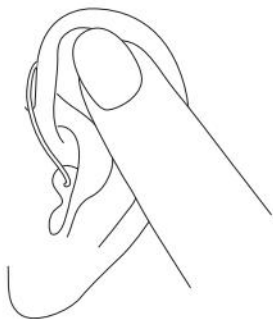

11. Connectivity overview

The illustration below shows the connectivity options available for your hearing aid.

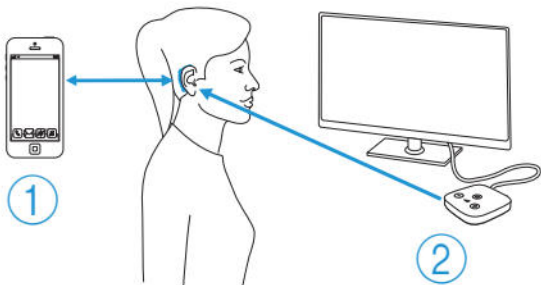

1 Bluetooth® enabled device, e.g. cell phone

2 TV Connector connected to TV*

* The TV Connector can be connected to any audio source such as a TV, PC or hi-fi system.

12. Initial pairing

12.1 Initial pairing to a Bluetooth® enabled device


i It is only necessary to perform the pairing procedure once with each device featuring Bluetooth wireless technology. After the initial pairing, your hearing aids will connect automatically to the device. Initial pairing can take up to 2 minutes.

1.

On your device (e.g. a phone), ensure that Bluetooth® wireless technology is enabled and search for Bluetooth® enabled devices in the connectivity setting menu.

2.

Switch on both hearing aids. You now have 3 minutes to pair your hearing aids with your device.

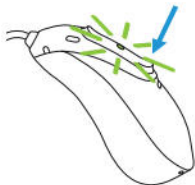

3.

Your device shows a list of Bluetooth® enabled devices. Select the hearing aid from the list to simultaneously pair both hearing aids. A beep confirms successful pairing.

- i** For more information about pairing instructions for Bluetooth wireless technology, specific to some of the most popular phone manufacturers, go to: hearingaids.sennheiser-hearing.com/BluetoothSetup


12.2 Connecting to the device

After your hearing aids have been paired to your device, they will automatically connect again when switched on.

- i** The connection will be maintained as long as the device remains ON and within range.
- i** Your hearing aid can be connected to up to two devices and paired to up to eight devices.

13. Phone calls

Your hearing aids connect directly with Bluetooth® enabled phones. When paired and connected to your phone, you will hear notifications, incoming call notification and the caller's voice directly in your hearing aids. Phone calls are "hands free", this means your voice is picked up by the hearing aid's microphones and transmitted back to the phone.

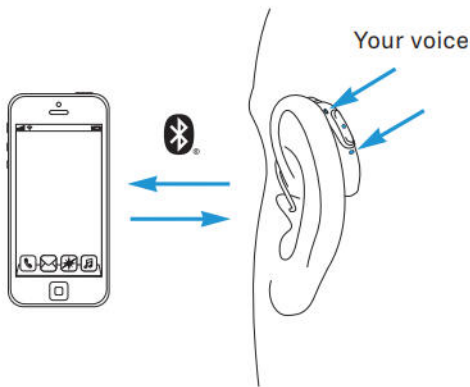


13.1 Making a call

Enter the phone number and press the dial button. You will hear the dialing tone through your hearing aids. Your voice is picked up by the hearing aid microphones and transmitted to the phone.

13.2 Accepting a call


When receiving a call, in-coming call notification will be heard in the hearing aids (e.g. ring tone or voice notification).

The call can be accepted by tapping the top of your ear twice, a short press on the upper or lower part of the multi-function button on the hearing aid (less than 2 seconds) or directly on your phone.

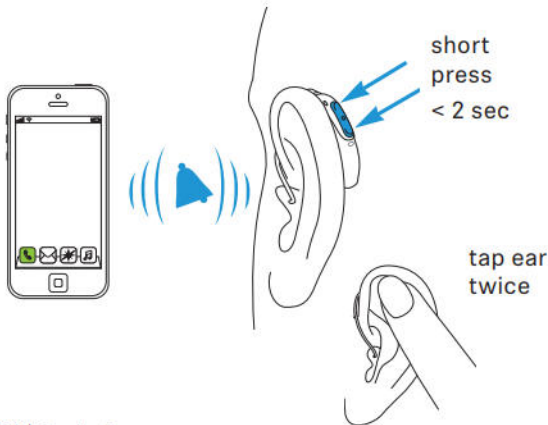


13.3 Ending a call

A call can be ended by a long press on the upper or lower part of the multi-function button on the hearing aid (more than 2 seconds) or directly on your phone. The phone call can also be ended by tapping the top of your ear twice.

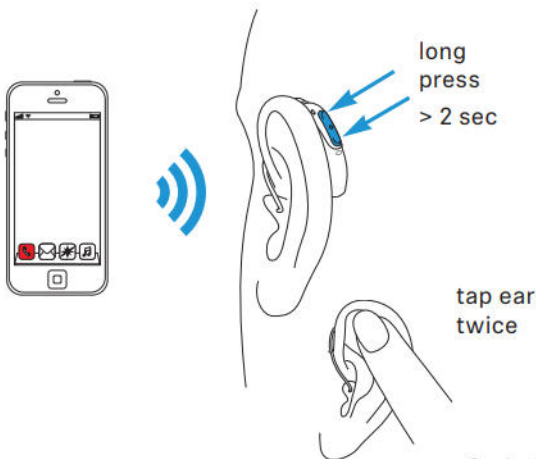

13.4 Rejecting a call

An incoming call can be rejected by a long press on the upper or lower part of the multi-function button on the hearing aid (more than 2 seconds) or directly on your phone.

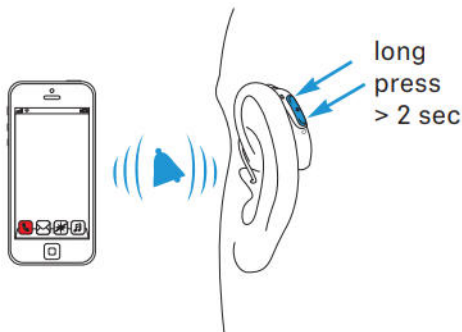


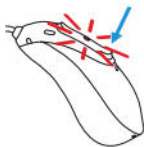
14. Flight mode


Your hearing aids operate between 2.4 GHz–2.48 GHz frequency range. When flying some operators require all devices to be switched into flight mode. Entering flight mode will not disable normal hearing aid functionality, only the Bluetooth connectivity functions.


14.1 Enter flight mode


To disable the wireless function and enter flight mode in each hearing aid:

-
1.  If the hearing aid is turned off, proceed to Step 2. If the hearing aid is on, turn it off by pressing and holding the lower part of the button for 3 seconds, until the indicator light turns solid red for 3 seconds, signifying that the hearing aid is switching off.



-
2.  Press and hold the lower part of the button. The green indicator light blinks when the hearing aid starts up.

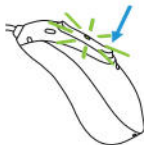

-
3.  Keep pressing the button for 7 seconds, until the indicator light is solid orange, then release the button. The solid orange light indicates the hearing aid is in flight mode.




In flight mode, your hearing aid cannot connect directly to your phone.

14.2 Exit flight mode

To enable the wireless function and exit flight mode in each hearing aid:

Switch the hearing aid off and then on again – refer to chapter 9.

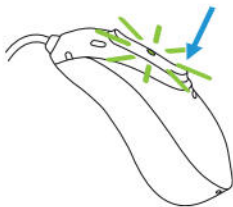

15. Rebooting your hearing aid

Use this technique if your hearing aid is in an error state. This will not remove or delete any program or settings.

1.

Press the lower part of the multi-function button for at least 15 seconds. It does not matter if the hearing aid is switched on or off before you start pressing the button. There is no light or acoustic notification at the end of the 15 seconds.

2.

Place the hearing aid in the power-connected charger and wait until the indicator light blinks green. This can take up to 30 seconds. The hearing aid is now ready to use.

16. Easy Line Remote™ app overview

Compatibility information

The Easy Line Remote app can be used on:

- Phones with **Bluetooth** Low Energy (LE) capability
- iPhone® 5s and newer running iOS® Version 12.0 or newer
- Google Mobile Services (GMS) certified Android™ devices supporting Bluetooth 4.2 and Android™ OS 7.0 or newer

Please find the Instruction for Use on hearingaids.sennheiser-hearing.com/AppUserGuide or scan the code.



Install the Easy Line Remote app for iPhone users.



Install the Easy Line Remote app for users of smartphones operating on Android OS.



iPhone® is a trademark of Apple Inc., registered in the U.S. and other countries.

iOS® is a registered trademark or trademark of Cisco Systems, Inc. and/or its affiliates in the United States and certain other countries.

Android™ is a trademark of Google LLC.

17. Environmental conditions

The product is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in these user guides.

Please ensure to use, charge, transport and store the hearing aids and the charger according to the following conditions:

Sennheiser Sonite R

| | Use | Charging | Transport | Storage |
|-----------------------------|--------------------------------|--------------------------------|---------------------------------|---------------------------------|
| Temperature | | | | |
| Maximum range | +5 to +40° C (41 to 104° F) | +5 to +40° C (41 to 104° F) | -20 to +60° C (-4 to 140° F) | -20 to +60° C (-4 to 140° F) |
| Recommended range | +5 to +40° C (41 to 104° F) | +7 to +32° C (45 to 90° F) | 0 to +20° C (32 to 68° F) | 0 to +20° C (32 to 68° F) |
| Humidity | | | | |
| Maximum range | 0% to 85% (non condensing) | 0% to 85% (non condensing) | 0% to 93% | 0% to 93% |
| Recommended range | 0% to 60% (non condensing) | 0% to 60% (non condensing) | 0% to 60% | 0% to 60% |
| Atmospheric pressure | 500 to 1060 hPa | 500 to 1060 hPa | 500 to 1060 hPa | 500 to 1060 hPa |

Sennheiser S Charger R

| | Use | Transport | Storage |
|-----------------------------|--------------------------------|---------------------------------|---------------------------------|
| Temperature | | | |
| Maximum range | +5 to +40° C (41 to 104° F) | -20 to +60° C (-4 to 140° F) | -20 to +60° C (-4 to 140° F) |
| Recommended range | +7 to +32° C (45 to 90° F) | 0 to +20° C (32 to 68° F) | 0 to +20° C (32 to 68° F) |
| Humidity | | | |
| Maximum range | 0% to 85% (non condensing) | 0% to 70% | 0% to 70% |
| Recommended range | 0% to 60% (non condensing) | 0% to 60% | 0% to 60% |
| Atmospheric pressure | 500 to 1060 hPa | 500 to 1060 hPa | 500 to 1060 hPa |

These hearing aids are classified as IP68. This means that they are water and dust resistant and designed to withstand daily life situations. They can be worn in the rain but should not be fully submerged in water or used when taking a shower, swimming or other water activities. These hearing aids should never be exposed to chlorinated water, soap, salt water or other liquids with a chemical content.

This charger is classified as IP22. This means that the charger is protected from vertically falling drops of water when the charger is tilted up to 15 degrees from the horizontal. IP22 is satisfied when the charger lid is closed.

Please ensure to charge the hearing aids regularly during long-term storage.

If the device has been subject to storage and transport conditions outside of the recommended use conditions, wait 15 minutes before starting the device.

18. Care and maintenance

Expected Service Life:

The hearing aids and the compatible charger have the expected service life of five years. The devices are expected to remain safe for use for this duration.

Commercial service period:

Routine and diligent care of your hearing aids and charger will contribute to the outstanding performance for the duration of the expected service life of your devices.

The manufacturer will provide a minimum of a five year repair service period after the respective hearing aid, charger and essential components have been phased out of the product portfolio.


Please use the following specifications as a guideline. For further information regarding product safety, see chapter 24.2.

General information

Before using hair spray or applying cosmetics, you should remove your hearing aid from your ear, because these products may damage it.

Your hearing aids are resistant to water, sweat and dust under the following conditions:

- After exposure to water, sweat or dust, the hearing aid is cleaned and dried.
- The hearing aid is used and maintained as described in this user guide.

 Always make sure that the hearing aids and charger are dry and clean before charging.

Daily

Hearing aid: Inspect the earpiece for earwax and moisture deposit and clean the surfaces with a lint-free cloth. Never use cleaning agents such as household detergents, soap, etc. for cleaning your hearing aid. It is not recommended to rinse with water. If you need to clean your hearing aid intensively, ask your hearing care professional for advice and information on filters or drying methods.

Charger: Make sure that charger inserts are clean and clean the surfaces with a lint-free cloth. Never use cleaning agents such as household detergents, soap, etc. for cleaning your charger.

Weekly

Hearing aid: Clean the earpiece with a soft, damp cloth or with a special cleaning cloth for hearing aids. For more in-depth maintenance instructions, please see your hearing care professional. Clean the charging contacts on the hearing aid with a soft, damp cloth.

Chargers: Remove dust or dirt from the charger inserts with a lint-free cloth.



Always make sure that the charger is disconnected from the power mains before cleaning.

19. Exchanging the earwax protection


Your hearing aid is equipped with an earwax protection to protect the speaker from earwax damage.

Please check the earwax protector regularly and exchange it if it looks dirty or if the hearing aid volume or sound quality is reduced. The earwax protection should be replaced every four to eight weeks.

19.1 Removing the earpiece from the speaker


1.

Remove the earpiece from the speaker by holding the speaker in one hand and the earpiece in the other.

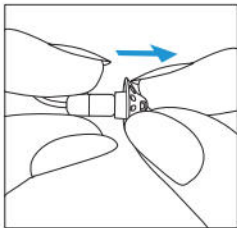


2.

Gently pull off the earpiece to remove.

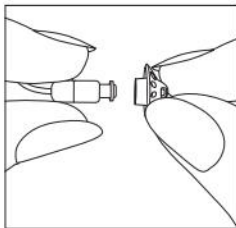

3.

Clean the speaker with a lint-free cloth.

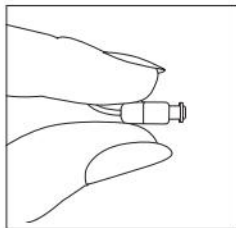


19.2 Exchanging the earwax protection with CeruStop

To replace the earwax protector on the receiver, only use the wax filter tool that was recommended by your hearing care professional (see page 3).


1.

Insert the removal side of the exchange tool into the used earwax protector. The shaft of the holder should touch the rim of the earwax protector.

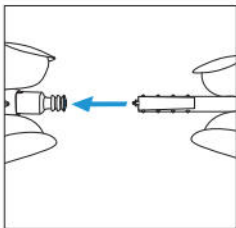


2.

Carefully pull and remove the earwax protector straight out of the speaker. Do not twist the earwax protector when removing.

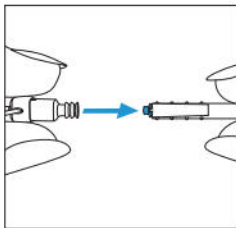


3.

To insert the new earwax protector, gently push the insertion side of the exchange tool straight into the hole of the speaker until the outer ring is perfectly aligned.

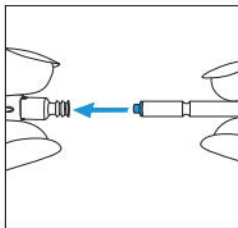

4.


Pull the tool straight out. The new earwax protector will remain in place. Now attach your earpiece to the speaker.

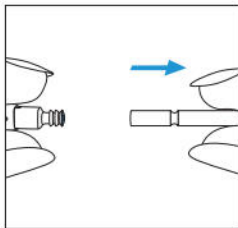

19.3 Attaching the earpiece to the speaker


1.

Hold the speaker in one hand and the earpiece in the other.

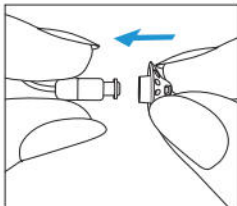


2.

Slide the earpiece over the speaker sound outlet.

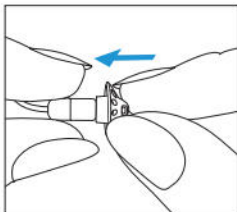

3.

The speaker and the earpiece should fit perfectly together.

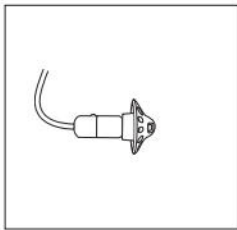

20. Service and warranty

Local warranty

Please ask the hearing care professional where you purchased your hearing aids and charger about the terms of the local warranty.

International warranty

The manufacturer offers a one year limited international warranty, valid from the date of purchase. This limited warranty covers manufacturing and material defects in the hearing aid and charger respectively, but not accessories such as tubes, earpieces, external receivers. The warranty only comes into force if a proof of purchase is shown.

The international warranty does not affect any legal rights that you might have under applicable national legislation governing sale of consumer goods.

Warranty limitation

This warranty does not cover damage from improper handling or care, exposure to chemicals or undue stress. Damage caused by third parties or non-authorized service centers renders the warranty null and void. This warranty does not include any services performed by a hearing care professional in their office.

Serial numbers

Hearing aid left side: _____

Hearing aid right side: _____

Charger: _____

Date of purchase: _____

Authorized hearing care
professional (stamp/signature):

21. Compliance information

Europe:

Declaration of Conformity for the hearing aid

Hereby the manufacturer declares that this product meets the requirements of the Medical Device Regulation (EU) 2017/745 as well as the Radio Equipment Directive 2014/53/EU.

The full text of the EU Declaration of Conformity can be obtained from the manufacturer:

www.sonova.com/en/certificates
(Sonova worldwide location).

Australia/New Zealand:



R-NZ

Indicates a device's compliance with applicable Radio Spectrum Management's (RSM) and Australian Communications and Media Authority (ACMA) regulatory arrangements for the legal sale in New Zealand and Australia.

The compliance label R-NZ is for radio products supplied in the New Zealand market under conformity level A1.

The wireless models listed on page 2 are certified under:

Sennheiser Sonite R

Sennheiser Sonite R Loaner

USA **FCC ID:** KWC-MRP

Canada **IC:** 2262A-MRP

Notice 1:

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions:

- 1) this device may not cause harmful interference, and
- 2) this device must accept any interference received, including interference that may cause undesired operation.

Notice 2:

Changes or modifications made to this device not expressly approved by manufacturer may void the FCC authorization to operate this device.

Notice 3:

This device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules and ICES-003 of Industry Canada.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This device generates,

uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this device does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Charger

- Please keep at least 20 cm separation distance between the charger and the human body during operation.

Radio information of your wireless hearing aid

| | |
|---------------------|-----------------------------------|
| Antenna type | Resonant loop antenna |
| Operation frequency | 2.4 GHz – 2.48 GHz |
| Modulation | GFSK, Pi/4 DPSK, GMSK |
| Radiated power | < 2.5 mW |
| Bluetooth® | |
| Range | ~1 m |
| Bluetooth® | 4.2 Dual-Mode |
| Profiles supported | HFP (Hands-free profile), A2DP |

Electromagnetic emissions

Manufacturer's Declaration: The devices are intended for use in the electromagnetic environments listed below. The user should assure that they are used in such environments.

| Emissions Test | Compliance | Electromagnetic environment guidance |
|--------------------------|------------|---|
| RF emissions CISPR 11 | Group 1 | The device generates or uses RF energy only for their internal functioning. Therefore, the RF emissions are very low and are not likely to cause any interference to nearby electronic equipment. |
| RF emissions CISPR 11 | Class B | The device is suitable for use in domestic establishment and in establishment directly connected to a low voltage power supply network which supplies buildings used for domestic purposes. |

Electromagnetic immunity

| Immunity Test | IEC60601-1-2 Test Level | Compliance Level |
|---|---|---|
| Electrostatic Discharge (ESD) IEC 61000-4-2 | +/- 8 kV contact +/- 2 kV , +/- 4 kV, +/- 8 kV, +/- 15 kV air | +/- 8 kV contact +/- 2 kV , +/- 4 kV, +/- 8 kV, +/- 15 kV air |
| Radiated RF Electromagnetic Field Immunity Test IEC 61000-4-3 | 10 V/m 80 MHz – 2.7 GHz 80% AM at 1 kHz | 10 V/m 80 MHz – 2.7 GHz 80% AM at 1 kHz |

| Immunity Test | IEC60601-1-2 Test Level | Compliance Level |
|---|--|---------------------------|
| Proximity Magnetic Field IEC 61000-4-3 | 9 V/m; 710, 745, 780 MHz; | 9 V/m; 710, 745, 780 MHz; |
| | Pulse modulation | Pulse modulation |
| | 18 Hz | 18 Hz |
| | 9 V/m; | 9 V/m; |
| | 5240, 5500, 5785 MHz; | 5240, 5500, 5785 MHz; |
| | Pulse modulation 217 Hz | Pulse modulation 217 Hz |
| | 27 V/m; 385 MHz; | 27 V/m; 385 MHz; |
| Pulse modulation 18 Hz | Pulse modulation 18 Hz | |
| 28 V/m; 450 MHz FM +/- 5 kHz deviation; | 28 V/m; 450 MHz FM +/- 5 kHz deviation; | |
| 1 kHz sine | 1 kHz sine | |
| 28 V/m; 810, 870, 930 MHz; Pulse modulation | 28 V/m; 810, 870, 930 MHz; | |
| 18 Hz | Pulse modulation 18 Hz | |
| 28 V/m; 1720, 1845, 1970, 2450 MHz; | 28 V/m; 1720, 1845, 1970, 2450 MHz; | |
| Pulse modulation | Pulse modulation | |
| 217 Hz | 217 Hz | |

| Immunity Test | IEC60601-1-2 Test Level | Compliance Level |
|--|--------------------------------|--------------------------|
| Rated power frequency magnetic fields IEC 61000-4-8 | 30 A/m 50 Hz or 60 Hz | 30 A/m 50 Hz or 60 Hz |

22. Information and description of symbols



With the CE symbol, manufacturer confirms that this product – including accessories – meets the requirements of the Medical Devices Regulation (EU) 2017/745 as well as the Radio Equipment Directive 2014/53/EU. The numbers after the CE symbol correspond to the code of certified institutions that were consulted under the above-mentioned regulation.



This symbol indicates that the products described in these user instructions adhere to the requirements for an applied part of Type B of EN 60601-1. The surface of the hearing aid is specified as applied part of Type B.



Indicates the medical device manufacturer, as defined in Medical Device Regulation (EU) 2017/745.



Indicates the date when the medical device was manufactured.



Indicates the Authorized representative in the European Community. The EC REP is also the importer to the European Union.



This symbol indicates that it is important for the user to read and take into account the relevant information in these user guides.



This symbol indicates that it is important for the user to pay attention to the relevant warning notices in this user guide.



This symbol indicates that it is important for the user to pay attention to the relevant warning notices related to batteries in these user guides.



Important information for handling and product safety.



This symbol confers that the electromagnetic interference from the device is under limits approved by the US Federal Communications Commission.



Indicates a device's compliance with applicable Radio Spectrum Management's (RSM) and Australian Communications and Media Authority (ACMA) regulatory arrangements for the legal sale in New Zealand and Australia.



The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by manufacturer is under license. Other trademarks and trade names are those of their respective owners.



Japanese mark for certified radio equipment.

SN

Indicates the manufacturer's serial number so that a specific medical device can be identified.

REF

Indicates the manufacturer's catalogue number so that the medical device can be identified

MD

Indicates that the device is a medical device.



An indication that electronic instructions for use are available.

IP68

IP Rating = Ingress Protection Rating. IP68 rating indicates that the hearing aid is water and dust resistant. It survived continuous immersion in 1 meter of water for 60 minutes and 8 hours in a dust chamber as per the IEC60529 standard.

IP22

IP Rating = Ingress Protection Rating. IP22 rating indicates that the device is protected against solid foreign objects of 12,5 mm diameter and greater and is additionally protected against vertically falling water drops when enclosure tilted up to 15°.



Indicates the temperature limits to which the medical device can be safely exposed.



Indicates the range of humidity to which the medical device can be safely exposed.



Indicates the range of atmospheric pressure to which the medical device can be safely exposed.



During transportation keep dry



The symbol with the crossed-out garbage bin is to make you aware that this hearing aid as well as the charger may not be thrown away as normal household waste. Please dispose of old or unused hearing aids and charger at waste disposal sites intended for electronic waste, or give your hearing aid and charger to your licensed professional for disposal. Proper disposal protects the environment and health.



This symbol indicates a device that is not safe to use in magnetic resonance (MR) environment (e.g. during an MRI scan).



With the CE symbol, manufacturer confirms that this product – including accessories – meets the requirements of the Medical Devices Regulation MDR 2017/745 as well as the Radio Equipment Directive 2014/53/EU.



With the UK Conformity Assessed mark, manufacturer confirms that the product is conform to applicable Great Britain legislations.

Symbols applicable to the European power supply only



Power supply with double isolation.



Device designed for indoor use only.



Safety isolating transformer, short circuit proof.

Feature symbols



Advanced technology



Bluetooth connectivity



Speech enhancement



Exceptional durability



Hands-free calls & music

23. Troubleshooting

Problem: Hearing aid not functioning

| | |
|----------------------------|---|
| Speaker/earpiece blocked | Clean speaker opening/earpiece |
| Hearing aid switched off | Press lower part of multi-function button for 3 seconds (chapter 9) |
| Battery fully discharged | Charge hearing aid (chapter 4) |
| Hearing aid in error state | Reboot the hearing aid (chapter 16) |

Problem: Hearing aid whistles

| | |
|---|--|
| Hearing aid not inserted correctly into ear | Insert hearing aid correctly (chapter 6) |
| Cerumen in ear canal | Contact your ENT/GP or hearing care professional |

Problem: Hearing aid too loud

| | |
|-----------------|---|
| Volume too loud | Decrease volume if volume control available (chapter 8) |
|-----------------|---|

Problem: Hearing aid not loud enough or distorted

| | |
|--------------------------|---|
| Volume too low | Increase volume if volume control available (chapter 8) |
| Low battery | Charge hearing aid (chapter 4) |
| Speaker/earpiece blocked | Clean speaker opening/earpiece |
| Hearing has changed | Contact your hearing care professional |

Problem: Hearing aid plays two beeps

| | |
|----------------------------|--------------------------------|
| Indication for low battery | Charge hearing aid (chapter 4) |
|----------------------------|--------------------------------|

Problem: Hearing aid will not switch on

| | |
|--|--|
| Battery fully discharged | Charge hearing aid (chapter 4) |
| Button was inadvertently deactivated by pressing the button longer than 15 seconds | Perform step 2 of the rebooting process (chapter 15) |

Please check <https://www.sennheiser-hearing.com/soniteR/support> for further information.

Problem: Indicator light on hearing aid does not light up when hearing aid placed in charging slots

| | |
|---|---|
| Hearing aid not inserted correctly into charger | Insert hearing aid correctly into charger (chapter 4) |
| Hearing aid battery fully discharged | Wait three hours after placing hearing aid in charger, regardless of LED behavior |

Problem: Indicator light on hearing aid is solid red while hearing aid is in the charger

| | |
|---|--|
| Hearing aid is out of its operating temperature range | Warm hearing aid up or cool it down. Operating temperature range is +5° to +40° Celsius (+41° to +104° Fahrenheit) |
| Faulty battery | Contact your hearing care professional |

Problem: Indicator light on hearing aid switches off when hearing aid is removed from charger

| | |
|-------------------------------|---------------------------------------|
| “Auto On” feature is disabled | Switch on the hearing aid (chapter 9) |
|-------------------------------|---------------------------------------|

Problem: Indicator light on hearing aid remains solid green after removal from charger

Indicator light on hearing aid was red when hearing aid was placed into the charger

Reboot the hearing aid (chapter 15)

Problem: Battery does not last the whole day

Shortened battery life

Contact your hearing care professional. A battery replacement may be required.

Problem: Phone call function does not work

Hearing aid is in flight mode

Switch hearing aid off and then on again (chapter 14.2)

Hearing aid not paired to the phone

Pair hearing aid to the phone (chapter 12)

- i** If the problem persists, contact your hearing care professional for assistance.

Please check <https://www.sennheiser-hearing.com/soniteR/support> for further information.

Problem: Charging does not start (but indicator light around the charger USB port lights up green when the charger is connected to the power outlet.)

Hearing aid not inserted correctly into charger

Insert hearing aid correctly into charger (chapter 4)

24. Important safety information

Please read the relevant safety information and limitations-of-use information on the following pages before using your hearing aid.

Intended use

Hearing aid: The hearing aid is intended to amplify and transmit sound to the ear and thereby compensate for impaired hearing.

The Tinnitus Balance software feature is intended for use by people with tinnitus who also desire amplification. It provides supplementary noise stimulation which can help defocus the user's attention from their tinnitus.

Charger: The charger is intended to charge the battery of the rechargeable hearing aid.

Intended patient population

Hearing aid: This device is intended for patients with unilateral and bilateral, mild to profound hearing loss from 8 years of age.

The Tinnitus Balance software feature is intended for patients with chronic tinnitus from 18 years of age.

Charger: The intended patient population is the user of the compatible hearing aids.

Intended user

Hearing aid: Intended for people with hearing loss using a hearing aid and their caregivers. A hearing care professional is responsible for adjusting the hearing aid.

Charger: Intended for people with hearing loss using the compatible rechargeable hearing aid and their caregivers, hearing care professional.

Indications

General clinical indications for the use of hearing aids and Tinnitus Balance software feature are:

Presence of hearing loss

- Uni- or bilateral
- Conductive, sensorineural or mixed
- Mild to profound

Presence of a chronic tinnitus (only applicable for devices which provide the Tinnitus Balance software feature)

Charger: Please note, indications are not derived from the charger, but from the compatible hearing aids.

Contraindications

General clinical contra-indications for the use of hearing aids and Tinnitus Balance software feature are:

- Hearing loss is not in the fitting range of the hearing aid (i.e. gain, frequency response)
- Acute tinnitus (< three months after it occurs)
- Deformity of the ear (i.e. closed ear canal; absence of the auricle)
- Neural hearing loss (i.e. retro-cochlear pathologies such as absent / non-viable auditory nerve)

Charger: Please note, contraindications are not derived from the charger, but from the compatible hearing aids.

The primary criteria for the referral of a patient for a medical or other specialist opinion and / or treatment are as follows:

- Visible congenital or traumatic deformity of the ear;
- History of active drainage from the ear in the previous 90 days;
- History of sudden or rapidly progressive hearing loss in one or both ears within the previous 90 days;

- Acute or chronic dizziness;
- Audiometric air-bone gap equal to or greater than 15 dB at 500 Hz, 1000 Hz and 2000 Hz;
- Visible evidence of significant cerumen accumulation or a foreign body in the ear canal;
- Pain or discomfort in the ear;
- Abnormal appearance of the eardrum and ear canal such as:
 - Inflammation of the external auditory canal
 - Perforated eardrum
 - Other abnormalities which the hearing care professional believes are of medical concern

Special care should be exercised in selecting and fitting a hearing aid whose maximum sound pressure level exceeds 132 decibels (dB) because there may be risk of impairing the remaining hearing of the hearing aid user. (This provision is required only for those hearing aids with a maximum sound pressure capability greater than 132 dB).

The hearing care professional may decide that referral is not appropriate or in the best interests of the patient when the following applies:

- When there is sufficient evidence that the condition has been fully investigated by a medical specialist and any possible treatment has been provided;
- The condition has not worsened or changed significantly since the previous investigation and / or treatment.
- If the patient has given their informed and competent decision not to accept advice to seek a medical opinion, it is permissible to proceed to recommend appropriate hearing aid systems subject to the following considerations:
 - The recommendation will not have any adverse effects on the patients health or general well being;
 - The records confirm that all necessary considerations about the patient's best interests have been made.

If legally required, the patient has signed a disclaimer to confirm that the referral advice has not been accepted and that it is an informed decision.

Clinical benefit

Hearing aid: Improvement of speech understanding

Tinnitus Balance software feature: Tinnitus feature provides supplementary noise stimulation which can help defocus the user's attention from their tinnitus.

Charger: The Charger itself does not provide a direct clinical benefit. An indirect clinical benefit is provided by enabling the parent device (rechargeable hearing aids) to achieve their intended purpose.

Side effects

Physiological side-effects of hearing aids like tinnitus, dizziness, wax build up, too much pressure, sweating or moisture, blisters, itching and/or rashes, plugged or fullness and their consequences like headache and/ or ear pain, may be resolved or reduced by your hearing care professional.

Conventional hearing aids have the potential to expose patients to higher levels of sound exposure, which might result in threshold shifts in the frequency range affected by acoustic trauma.

Charger: Please note, side effects are not derived from the charger, but from the compatible hearing aids.

The hearing aid is suitable for the home healthcare environment and due to their portability it may happen that it is used in professional healthcare facility environment like physician offices, dental offices, etc.

A hearing aid will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions. Infrequent use of a hearing aid does not permit a user to attain full benefit from it. The use of a hearing aid is only part of hearing habilitation and may need to be supplemented by auditory training and instruction in lipreading.

Feedback, poor sound quality, too loud or too soft sounds, inadequate fitting or problems when chewing or swallowing, can be resolved or improved during the fine-tuning process at the fitting procedure by your hearing care professional.







Any serious incident that has occurred in relation to the hearing aid, should be reported to the manufacturer representative and the competent authority of the state of residence. The serious





incident is described as any incident that directly or indirectly led, might have led or might lead to any of the following:

- a) the death of a patient, user or other person
- b) the temporary or permanent serious deterioration of a patient's, user's or other person's state of health
- c) a serious public health threat

To report an unexpected operation or event, please contact the manufacturer or a representative.


24.1 Hazard warnings


-  The rechargeable hearing aids contain lithium-ion batteries that can be taken onto airplanes as carry-on luggage. The hearing aids cannot be placed in the checked-in baggage and can only be placed in the carry-on luggage.
-  Your hearing aids operate between 2.4 GHz -2.48 GHz frequency range. When flying please check if flight operator requires devices to be switched into flight mode, see chapter 14.
-  The hearing aids (specially programmed for each hearing loss) must only be used by the intended person. They should not be used by any other person as they could damage hearing.
-  Changes or modifications to the hearing aid and the charger that were not explicitly approved by Sonova AG are not permitted. Such changes may damage your ear or the hearing aid.
-  Do not use the hearing aids and charging accessories in explosive areas (mines or industrial areas with danger of explosions, oxygen rich environments or areas where flammable anesthetics are handled) or where electronic equipment is prohibited. The device is not ATEX certified.
-  If you feel pain in or behind your ear, if it is inflamed or if skin irritation and accelerated accumulations of earwax occur, please check with your hearing care professional or physician.


-  Titanium shells of custom hearing aids and earpieces can contain small quantities of acrylates from adhesives. People who are sensitive to such substances may potentially experience an allergic skin reaction. Please stop using the device immediately if such a reaction occurs and check with your hearing care professional or physician.
-  In very rare cases, the dome can remain in your ear canal when removing the hearing tube from the ear. In the unlikely case that the dome does get stuck in your ear canal, it is strongly recommended to see a physician for safe removal.
-  Hearing programs in the directional microphone mode reduce background noises. Please be aware that warning signals or noises coming from behind, e. g. cars, are partially or entirely suppressed.
-  This hearing aid is not for children below 36 months. The usage of this device by children and individuals with cognitive impairment should be supervised at all times to ensure their safety. The hearing aid is a small device and contains small parts. Do not leave children and individuals with cognitive impairment unsupervised with this hearing aid. If swallowed, consult a physician or hospital immediately as the hearing aid or its parts can cause choking!







 The following is only applicable for persons with active implantable medical devices (i.e. pacemakers, defibrillators, etc.):

- Keep the wireless hearing aid and the charging device at least 15 cm (6 inches) away from the active implant. If you experience any interference, do not use the wireless hearing aids and contact the manufacturer of the active implant. Please, note that interference can also be caused by power lines, electrostatic discharge, airport metal detectors etc.
- Keep magnets (i.e battery handling tool, EasyPhone magnet, etc.) at least 15 cm (6 inches) away from the active implant.

 Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

 Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

 Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the hearing aids and charger, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

-  Hearing aids should not be fitted with domes / wax protection systems when used by clients with perforated eardrums, inflamed ear canals or otherwise exposed middle ear cavities. In these cases, we recommend the use of a custom earpiece. In the unlikely case that any part of this product should remain in the ear canal, it is strongly recommended to see a medical specialist for safe removal.
-  The hearing aids have to be dry before charging. Otherwise the charging reliability cannot be guaranteed.
-  Avoid strong physical impacts to the ear when wearing a hearing aid with customized earpiece. The stability of customized earpieces is designed for normal use. A strong physical impact to the ear (e.g. during sports) may cause the customized earpiece to break. This may lead to perforation of the ear canal or eardrum.
-  After mechanical stress or shock to the customized earpiece, please ensure that it is intact before placing it in the ear.
-  The majority of hearing aid users have a hearing loss for which a deterioration in hearing is not to be expected with regular use of a hearing aid in everyday situations. Only a small group of hearing aid wearers with hearing loss, may be at risk of deterioration in hearing after a long period of use.
-  Do not remove receiver cables from the hearing instruments. If removal or exchange is required it is recommended to contact the hearing care professional.



Patients implanted with magnetic externally programmable CSF shunt valves may have a risk of experiencing an unintended change in their valve setting when exposed to strong magnetic fields.

The receiver (loudspeaker) of hearing aids, the Lyric SoundLync, the Lyric programming wand and the MiniControl contain static magnets. Keep a distance of 2 inches (approx. 5 cm) or more between the magnets and the site of the implanted shunt valve.



WARNING: The hearing aids use a lithium button/coin battery. These batteries are hazardous and can cause severe or fatal injuries in 2 hours or less if they are swallowed or placed inside any part of the body, whether the battery is used or new! Keep out of the reach of children, individuals with cognitive impairment, or pets. If you suspect that a battery was swallowed or placed inside any part of the body, immediately consult your physician without any delay!



Your hearing aid(s) contain lithium-ion batteries with a Watt hour rating <20Wh which have been tested according to UN 38.3 of the "UN Manual of Tests and Criteria" and should be shipped in accordance with all the rules and regulations on the safe shipment of lithium-ion batteries.





For charger, use only IEC 60950-1 *, IEC 62368-1 or IEC 60601-1 certified equipment with an output rating of 5 VDC. Min. 500 mA.



Do not touch the contacts of the charger when powered.

*Note: this standard may not be valid anymore in your country, please refer to IEC 62368-1.

 Cables and cords wrapped around the neck may cause strangulation. Do not leave the device and its components unattended with children, individuals with cognitive impairment or pets.

 Only authorized personnel are allowed to open the charging accessory due to possible electrical hazard.

24.2 Information on product safety

- i** Protect your hearing aid and charging accessories from heat and sunlight (never leave near a window or in the car). Never use a microwave or other heating devices to dry your hearing aid or charging accessories (due to risk of fire or explosion). Ask your hearing care professional about suitable drying methods.
- i** Do not place the hearing aids and charging accessories close to an induction cooking surface. Conductive structures inside the charging accessories could absorb inductive energy resulting in thermal destruction.
- i** The dome should be changed every three months or when it becomes stiff or brittle. This is to prevent the dome from detaching from the tube spout during insertion into or removal from the ear.
- i** Do not drop your hearing aid or charging accessories! Dropping onto a hard surface can damage your hearing aid or charging accessories.
- i** Charger and power supply must be protected against shocks. If the charger or power supply is damaged after a shock, the devices must not be used anymore.
- i** If you are not using your hearing aid for a long period of time, store it in a box with a drying capsule or in a well ventilated location. This allows moisture to evaporate from your hearing aid and prevents a potential impact on its performance.

i Special medical or dental examination including radiation described below, may adversely affect the correct functioning of your hearing aids. Remove and keep them outside the examination room/area before undergoing:

- Medical or dental examination with X-ray (also CT scan).
- Medical examinations with MRI/NMRI scans, generating magnetic fields.
Hearing aids don't need to be removed when passing security gates (airports etc.). If X-ray is used at all, it will be in very low doses, and will not affect the hearing aids.

i Only use approved charger described in this user guide for charging your hearing aids, otherwise the devices may be damaged.

i Make sure that you always completely dry your hearing aid after use. Store the hearing aid in a safe, dry and clean place.

i The hearing aids are water resistant and not waterproof. They are designed to withstand normal activities and occasional accidental exposure to extreme conditions. Never immerse your hearing aid in water! Phonak hearing aids are not specifically designed for extended periods of water submersion on a continual basis, that is worn in activities such as swimming or bathing. Always remove your hearing aid before these activities, as the hearing aid contains sensitive electronic parts. Never wash the microphone inputs. Doing so could cause it to lose its special acoustic features.

i Do not use your hearing aid or charger in areas where electronic equipment is prohibited.

Pat. sonova.com/en/intellectualproperty

Sennheiser is a registered trademark of Sennheiser electronic SE & Co. KG and is used under license by Sonova.



Sonova Deutschland GmbH
Max-Eyth-Strasse 20
70736 Fellbach-Oeffingen
Germany

Australian Sponsor:
Sonova Australia Pty Ltd
12 Inglewood Place,
Norwest NSW 2153
Australia



Sonova AG
Laubisrütistrasse 28
CH-8712 Stäfa,
Switzerland

**UK Responsible person
and UK Importer:**
Sonova UK Limited
Sonova House
Lakeside Drive, Centre Park
WA1 1RX Warrington
United Kingdom



0459

